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Arm Sling after Gilchrist

Suitable to immobilise the shoulder joint

Material: Polyester, terry cloth fabric, foam, velcro.

Indicators: Post-operatively following surgery on shoulder / conservative treatment for fractures of the proximal humerus and the scapula / post-luxation, post-contusion conditions

Application: Insert hand into the stocking opening equipped with a shoulder flap and pull on stocking. Avoid creating folds and ensure fingers can move freely and that shoulder flap is in correct position. / Take the shorter belt over the nape of the neck to the opposite side, along the thorax, and under the wrist, around the wrist and close the belt by hooking the sling on the fluffy section of the belt. A torsion of the belts must be prevented, ensure the terry cloth inner section is against the body. / Take the longer belt around the thorax to the back, pull under the arm supported in the stocking, and position hooks on the fluffy section on the back by creating a sling in the opposite direction. / Close the separate velcro loops above the wrist and the medial part of the upper arm by gathering the belts for both sling sections.

Precautionary measures: To avoid a feeling of constriction on the skin, above all in the area of the neck, a twisting of the belts must be prevented. The skin should make contact with the terry cloth section. The disposal of the device is the responsibility of the user. Local legal requirements should be observed. Store in a dry place.

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Allenspach Medical AG
TRIAMED GmbH & Co. KG
Hersteller: Omnimed AG, CH-4020 Basel