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EN

Immobilization is a basic principle in the treatment of trauma and injuries caused by overloading, as the healing process is significantly accelerated and consequential damages are prevented. Shoulder and arm braces support injured tissue structures (joints, ligaments, capsules, cartilage). Bort OmoXpress is a brace for shoulder immobilization that easily and safely fixes the shoulder joint while the arm is rotated inward and placed in an abduction position.

Instructions for use:

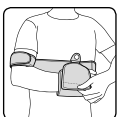
BORT OmoXpress brace for shoulder immobilization should be put on by a health professional

1. Place the belt around the torso under the breast and close the Velcro fastening.

2. Block the arm of the injured shoulder by using the strap in the distal region of the upper arm.


3. Place the hand strap around the wrist and close the Velcro fastening.

4. Close the rotary buckle between the waist belt and hand strap.

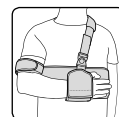5. The individual length of the waist belt can be adjusted using the two Velcro fastenings.

6. Check the patient's posture and, if necessary, correct the position of the brace.

Indications:

Pre-operative, post-operative, post-traumatic, for example in the case of sprains, contusions, dislocations, humerus fractures.

Contraindications:

Clavicle fractures, peripheral arterial occlusive disease (PAOD), lymphatic congestion, as well as unexplained swelling of the soft tissues in areas away from where the brace is applied, sensory disturbance and circulation disorders in the treated area, skin diseases in the treated area.

Material composition:

59 % polyurethane foam, 41 % polyamide

Important warning:

Do not re-use – this item is designed to be used for just one patient. If it is to be worn at night, impairment of blood circulation should be avoided. Should the hand or arm become numb, loosen the brace or remove if necessary. If symptoms persist, immediately consult your doctor.

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