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Instructions to User

Dear users, thank you very much for purchasing the Pulse Oximeter.

This Manual is written and compiled in accordance with the council directive MDD93/42/EEC for medical devices and harmonized standards. In case of modifications and software upgrades, the information contained in this document is subject to change without notice.

The Manual describes, in accordance with the Pulse Oximeter's features and requirements, main structure, functions, specifications, correct methods for transportation, installation, usage, operation, repair, maintenance and storage, etc. as well as the safety procedures to protect both the user and equipment. Refer to the respective chapters for details.

Please read the User Manual carefully before using this product. The User Manual which describes the operating procedures should be followed strictly. Failure to follow the User Manual may cause measuring abnormality, equipment damage and human injury. The manufacturer is NOT responsible for the safety, reliability and performance issues and any monitoring abnormality, human injury and equipment damage due to users' negligence of the operation instructions. The manufacturer's warranty service does not cover such faults.

Owing to the forthcoming renovation, the specific products you received may not be totally in accordance with the description of this User Manual. We would sincerely regret for that.

This product is medical device, which can be used repeatedly.

Warning

- ! Uncomfortable or painful feeling may appear if using the device ceaselessly, especially for the microcirculation barrier patients. It is recommended that the sensor should not be applied to the same finger for over 2 hours.
- ! For the special patients, there should be a more prudent inspecting in the placing process. The device can not be clipped on the edema and tender tissue.
- ! The light (the infrared is invisible) emitted from the device is harmful to the eyes, so the user and the maintenance man should not stare at the light.
- ! Testee can not use enamel or other makeup.
- ! Testee's fingernail can not be too long.
- ! Please refer to the correlative literature about the clinical restrictions and caution.
- ! This device is not intended for treatment.

The User Manual is published by our company. All rights reserved.

1. Safety

1.1 Instructions for Safe Operations

- ◇ Check the main unit and all accessories periodically to make sure that there is no visible damage that may affect patient's safety and monitoring performance about cables and transducers. It is recommended that the device should be inspected once a week at least. When there is obvious damage, stop using the device.
- ◇ Necessary maintenance must be performed by qualified service engineers ONLY. Users are not permitted to maintain it by themselves.
- ◇ The oximeter cannot be used together with devices not specified in User's Manual. Only the accessory that appointed or recommendatory by manufacture can be used with this device.
- ◇ This product is calibrated before leaving factory.

1.2 Warning

- ! Explosive hazard—DO NOT use the oximeter in environment with inflammable gas such as some ignitable anesthetic agents.

- ! DO NOT use the oximeter while the testee measured by MRI and CT.
- ! Please do not break the wristband, for fear it becomes out of use, or the unexpected drop of the device which is due to the looseness of the wristband in the process of using. Users who are allergic to the wristband is not recommended to use it.
- ! The person who is allergic to rubber can not use this device.
- ! The disposal of scrap instrument and its accessories and packings (including battery, plastic bags, foams and paper boxes) should follow the local laws and regulations.
- ! Please check the packing before use to make sure the device and accessories are totally in accordance with the packing list, or else the device may have the possibility of working abnormally.
- ! Please choose the accessories and probe which are approved or manufactured by the manufacturer, or else it may damage the device.
- ! Please choose the battery chargers which should be ensured compliance with the requirements of IEC 60601-1, or else it may damage the device.
- ! Please don't use the device in the course of charging.
- ! The device can only be matched with the compatible probe.
- ! Please don't measure this device with functional tester for the device's related information.

1.3 Attention

- Keep the oximeter away from dust, vibration, corrosive substances, explosive materials, high temperature and moisture.
- If the oximeter gets wet, please stop operating it.
- When it is carried from cold environment to warm or humid environment, please do not use it immediately.
- DO NOT operate keys on front panel with sharp materials.
- High temperature or high pressure steam disinfection of the oximeter is not permitted. Refer to User Manual in the relative chapter (7.1) for instructions of cleaning and disinfection.
- Do not have the oximeter immersed in liquid. When it needs cleaning, please wipe its surface with medical alcohol by soft material. Do not spray any liquid on the device directly.
- When cleaning the device with water, the temperature should be lower than 60°C.
- As to the fingers which are too thin or too cold, it would probably affect the normal measure of the patients' SpO₂ and pulse rate, please clip the thick finger such as thumb and middle finger deeply enough into the probe.
- The pulse oximeter can be used to adult or infant. Whether the device is used to adult or infant, it depends on the probe selected.
- The update period of data is less than 30 seconds, which is changeable according to different individual pulse rate.
- The waveform is normalized. Please read the measured value when the waveform on screen is equably and steady-going. Here this measured value is optimal value, and the waveform at the moment is the standard one.
- If some abnormal conditions appear on the screen during test process, pull out the finger and reinsert to restore normal use.
- The device has normal useful life for three years since the first electrified use.
- The device has an audible warning function, further information on this can be found in chapter 6.1.
- The device has a function for setting the limit values. If the measurement data exceeds or falls below the limit value, the device automatically emits an acoustic signal if you have switched on the function.
- The device has an acoustic warning function, which can be paused or switched off completely (factory setting). See chapter 6.1 for more information.
- The device may not work for all patients. If you are unable to achieve stable readings, discontinue use.

2. Overview

The pulse oxygen saturation is the percentage of HbO₂ in the total Hb in the blood, so-called the O₂ concentration in the blood. It is an important bio-parameter for the respiration. A number of diseases relating to respiratory system may cause the decrease of SpO₂ in the blood, furthermore, some other causes such as the malfunction of human body's self-adjustment, damages during surgery, and the injuries caused by some medical checkup would also lead to the difficulty of oxygen supply in human body, and the corresponding symptoms would appear as a consequence, such as vertigo, impotence, vomit etc. Serious symptoms might bring danger to human's life. Therefore, prompt information of patients' SpO₂ is of great help for the doctor to discover the potential danger, and is of great importance in the clinical medical field.

The Pulse Oximeter features in small volume, low power consumption, convenient operation and being portable. It is only necessary for patients to put one of his fingers into a probe for diagnosis, and a display screen will directly show the measured value of pulse oxygen saturation with the high veracity and repetition.

2.1 Features

- A. Operation of the product is simple and convenient.
- B. The product is small in volume, light in weight and convenient in carrying.
- C. Low power consumption

2.2 Major Applications and Scope of Application

The Pulse Oximeter can be used in measuring the pulse oxygen saturation and pulse rate through finger. The product is suitable for being used in family, hospital, oxygen bar, community healthcare, physical care in sports (It can be used before or after doing sports and it is not recommended to use the device during the process of having sport) and etc.

 The problem of overrating would emerge when the patient is suffering from toxicosis which caused by carbon monoxide, the device is not recommended to be used under this circumstance.

2.3 Environment Requirements

Storage Environment


- a. Temperature : -40°C~+60°C
- b. Relative humidity : ≤95%
- c. Atmospheric pressure : 500hPa~1060hPa

Operating Environment

- a. Temperature : 10°C~40°C
- b. Relative Humidity : ≤75%
- c. Atmospheric pressure : 700hPa~1060hPa

3. Principle

Principle of the Oximeter is as follows : An experience formula of data process is established taking use of Lambert Beer Law according to Spectrum Absorption Characteristics of Reductive Hemoglobin (Hb) and Oxyhemoglobin (HbO₂) in glow & near-infrared zones. Operation principle of the device is : Photoelectric Oxyhemoglobin Inspection Technology is adopted in accordance with Capacity Pulse Scanning & Recording Technology, so that two beams of different wavelength of lights can be focused onto human nail tip through perspective clamp finger-type sensor. Then measured signal can be obtained by a photosensitive element, information acquired through which will be shown on screen through treatment in electronic circuits and microprocessor.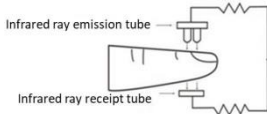

4. Technical Specifications

4.1 Main Performance

- A. SpO2 value display
- B. PR value and bar graph display
- C. Pulse waveform display
- D. Low-battery indication: low-battery indication appears when the battery voltage is too low to work.
- E. Adjustable screen brightness
- F. Prompt function for over-limit, probe off /finger-out and low battery.
- G. PR sound indication
- H. Memory function
- I. The data can be uploaded to the terminal equipment by wired mode.
- J. With clock function
- K. Charging function

4.2 Main Parameters

A. Measurement of SpO2

Measuring range: 0%~100%

Accuracy: When the SpO2 measuring range is 70%~100%, the permission of absolute error is $\pm 2\%$, Below 70% unspecified.

B. Measurement of pulse rate

Measuring range: 30bpm~250bpm

Accuracy: ± 2 bpm or $\pm 2\%$ (select larger)

C. Resolution

SpO2: 1%, Pulse rate: 1bpm

D. Measurement Performance in Weak Filling Condition

SpO2 and pulse rate can be shown correctly when pulse-filling ratio is 0.4%. SpO2 error is $\pm 4\%$, pulse rate error is ± 2 bpm or $\pm 2\%$ (select larger).

E. Resistance to surrounding light

The deviation between the value measured in the condition of man-made light or indoor natural light and that of darkroom is less than $\pm 1\%$.

F. Power supply requirement

3.6 V DC ~ 4.2V DC

G. Optical Sensor

Red light (wavelength is 660nm, 6.65mW)

Infrared (wavelength is 905nm, 6.75mW)


H. Adjustable prompt range

SpO2: 0%~100%

Pulse Rate: 0bpm~254bpm

5. Installation

5.1 Appearance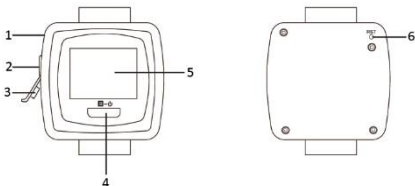

1. Charging indicator

4. Button

2. USB Port
3. USB plug

5. Screen
6. Reset hole

USB interface: connect with USB cable or SpO2 probe

Button: power on, pause acoustic signal, display clock, enter menu, menu operation.

RESET hole: reset button inside of it, to reset the device.

5.2 Interface introduction

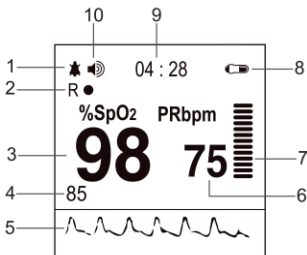
Figure 3 Measurement interface

- | | |
|-------------------------------------|---------------------|
| 1. Sound prompt | 6. PR value |
| 2. Record | 7. PR bar graph |
| 3. SpO2 value | 8. Battery power |
| 4. Over-limit prompt for SpO2 value | 9. Time |
| 5. PR waveform | 10. PR sound prompt |

5.3 SpO2 probe installation

Open the USB plug of the device, then insert the SpO2 probe interface into the USB port of the device.

5.4 Connection of USB cable

Open the USB plug of the device, insert the micro end of USB cable into the SpO2 probe interface, the other end into computer or power adapter.

5.5 Structure and accessories

A. Structure: main unit, SpO2 probe, USB cable, power adapter (optional).

B. Accessories: one SpO2 probe, one USB cable, one power adapter (optional), one User Manual.

Please check the device and accessories according to the list to avoid that the device can not work normally.

6. Operating

6.1 Measurement

A. Insert the finger into the probe as shown in Figure 4.

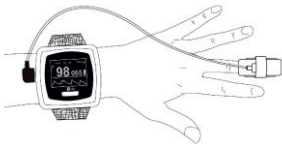
Fig. 4

(The appearance of actual probe may be different with the one shown as Figure 4, please refer to the actual probe.)

B. Press button to turn on the device, it displays the measurement interface.

C. Wait a few seconds, the device directly shows measurement result on the screen.

⚠ When inserting the finger, the light emitting from the sensor must be directly irradiated to the side of the fingernail.

⚠ During measuring, do not shake the finger and keep quiet, not move.

6.2 Measurement interface

A. In sound prompt “ON” state, when the sound prompt occurs, short press the button to pause the sound prompt, and it will resume automatically after about 60s.

Sound prompt includes over-limit prompt, low-battery prompt, probe off or finger out prompt.

B. If you want to turn off the sound prompt permanently, please set it in menu.

If there is no sound prompt, short pressing the button will directly enter clock interface, press again to exit.

6.3 Menu operation

Under the measurement interface, long press the button to enter the main menu interface as shown in Figure 5, sound, record, clock, system, etc. can be set, methods are as followings:

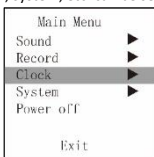
Fig. 5 Main menu

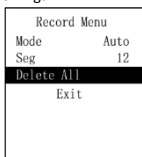
Fig. 7 Record menu

6.3.1 Sound prompt setting

Under main menu, short press the button to select “Sound”, then long press the button to enter its setting interface.

Short press the button to select the option to be adjusted, then long press the button to change the value.

“Direction”: direction, “up”: increase the value, “down”: decrease the value

“SpO2 HI(%)”: upper limit prompt for SpO2 over-limit

“SpO2 LO(%)”: lower limit prompt for SpO2 over-limit

“PR HI(BPM)”: upper limit prompt for PR over-limit

“PR LO(BPM)”: lower limit prompt for PR over-limit

“Prompt”: prompt for over-limit, “off”: close, “on”: open.

“Pulse Sound”: PR sound, “off”: close, “on”: open.

Lower limit can not exceed the upper limit, and the upper limit can not be lower than the lower limit when adjusting the values. SpO2 range: 0 % ~ 100 %, PR range: 0 ~ 254 bpm

The values displayed in Figure 6 are the initial values of over-limit prompt.

After setting, short press the button to select “Exit”, then long press the button to exit sound setting interface, and return to Main Menu interface.

6.3.2 Data storage

Under the main menu, short press the button to select “Record”, then long press the button to enter the Record Menu interface as shown in Figure 7. It indicates that the device is storing when the red dot “R•” in measurement interface flickers.

Short press the button to select the option to be adjusted, then long press the button to change the value.

“Mode”: record mode selection, including: “Auto” and “Manual” mode. Under “Manual” mode, select to turn on / off memory by “Record”.

Auto record: start recording after stable data appear, pull out the finger to finish recording a group of data (99 group of data at most), the total duration does not exceed 72 hours.

Manual record: store up to 24-hour data.

When the memory is full, it will display “Memory is full!”, then it will enter the standby mode after several seconds. When exiting the standby mode, it will display “Memory is full!” to give user a prompt, press the

button again to enter the measurement interface.

Note: under manual mode, when “Record” is “ON”, the device will prompt to clear the data stored last time. It will display “Recording” when there is no operation under record state for 30s, then it will enter energy saving mode after several seconds, long press the button to exit this mode; short press the button, it will display “Recording”.

Note: under data recording state, after the display screen turns off automatically, in order to save power, pulse sound indication will turn off automatically.

“Seg”: data segment.

After setting, short press the button to select “Exit”, long press the button to exit record menu and return to main menu.

“Delete All”: delete all records (auto record mode is shown as Figure 7).

Note: please upload data in time after recording, otherwise the data may be covered when the storage space is full.

Note: the historical data will be deleted once switching the record mode. Under recording state, the record mode can not be switched; under manual mode, the “Record” should be turned off before switching mode.

6.3.3 Clock setting

Under main menu, short press the button to select “Clock”, long press the button to enter its sub-menu as shown in Figure 8.

Set Time	no
Set Year	2019
Set Month	01
Set Day	01
Set Hour	03
Set Minute	00
Exit	

Fig. 8 Time setting interface

System Menu	
Hard.Ver.	2.0.0
Soft.Ver.	2.0.2
ID	user
Demo	off
Sound Volume	3
Brightness	1
Exit	

Fig. 9 System menu interface

Short press the button to select the option to be adjusted, then long press the button to change the value. The device adopts 24-hour clock.

“set time”: set the time, “yes”: to set the time, “no”: not set the time

“set year”: set the year

“set month”: set the month

“set day”: set the day

“set hour”: set the hour

“set minute”: set the minute

After setting, short press the button to select “Exit”, then long press the button to exit clock setting interface. If user changes the setting, current set time will be displayed, then press the button to return to the main menu; if the time is not set, it directly return to the main menu.

6.3.4 System setting and other options introduction

Under main menu, short press the button to select “System”, then long press the button to enter the interface as shown in Figure 9.

Short press the button to select the option to be adjusted, then long press the button to change the value.

“Hard.Ver.”: hardware version

“Soft.Ver.”: software version

“ID”: user name

“Demo”: set the Demo mode, “on”: turn on the Demo mode, “off”: turn off the Demo mode.

“Sound Volume”: set the sound volume, adjustable range: 1 ~ 3

“Brightness”: set the screen brightness, adjustable range: 1 ~ 4

After setting, short press the button to select “Exit”, then long press the button to exit the system menu and return to main menu.

6.3.5 Shutdown

Under the main menu interface, short press the button to select “Power off”, then long press the button to turn off the device.

6.3.6 Exit main menu

Under main menu, short press the button to select “Exit”, then long press the button to exit the main menu and return to the measurement interface.

6.4 Data upload

Connect the device to computer by the USB cable, upload the data after connecting with the PC software properly, refer to “Software operating instruction” for details.

Note: the PC software can be downloaded from our official website.

6.5 Charging

Power adapter can be selected to charge for the device.

It indicates that the device is charging when the indicator is orange, the charging is finished when the indicator turns to green.

6.6 Reset

Use a pointed and hard object (for example, a paper clip) to press the reset button inside of the RESET hole, to reset the device.

7. Maintain, Transportation and Storage

7.1 Cleaning and Disinfecting

The device must be turned off before cleaning, and it should not be immersed into liquid. Use 75% alcohol to wipe the device enclosure, and use liquid soap or isopropanol to wipe the watchband for disinfection, nature dry or clean it with clean and soft cloth. Do not spray any liquid on the device directly, and avoid liquid penetrating into the device.

7.2 Maintain

- Please clean and disinfect the device before using according to the User Manual (7.1).
- Please recharge the battery when the screen shows .
- Recharge the battery soon after the over-discharge. The device should be recharged every six months when it is no regular used. It can extend the battery life following this guidance.

7.3 Transportation and Storage


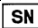
- The packed device can be transported by ordinary conveyance or according to transport contract. The device can not be transported mixed with toxic, harmful, corrosive material.
- The packed device should be stored in room with no corrosive gases and good ventilation. Temperature: -40°C~60°C; Humidity: ≤95%

8. Troubleshooting

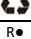Trouble	Possible Reason	Solution
The SpO2 and Pulse Rate can not be displayed normally	<ol style="list-style-type: none"> The finger is not properly positioned The patient's SpO2 is too low to be detected. 	<ol style="list-style-type: none"> Place the finger properly and try again. Try again; Go to a hospital for a diagnosis if you are sure the device works all right.
The SpO2 and Pulse Rate are not displayed stably	<ol style="list-style-type: none"> The finger is not placed inside deep enough. The finger is shaking or the patient is moving. 	<ol style="list-style-type: none"> Place the finger properly and try again. Let the patient keep calm
The device can not be turned on	<ol style="list-style-type: none"> The batteries are drained or almost drained. The device's malfunction 	<ol style="list-style-type: none"> Please recharge the battery Please contact the local service center.
The display is off suddenly	<ol style="list-style-type: none"> The device's malfunction. The battery is drained away or almost drained away. 	<ol style="list-style-type: none"> Please contact the local service center. Please recharge the battery

The device can not be used for full time after charge	The battery is not full charged. The battery is broken	1. Please recharge the battery 2. Please contact the local service center.
The battery can not be full charged even after 10 hours charging time.	The battery is broken	Please contact the local service center.

9. Key of Symbols

Symbol	Meaning	Symbol	Meaning
	Refer to instruction manual/booklet	PRbpm	Pulse rate (bpm)
	Type BF applied part	%SpO ₂	Pulse oxygen saturation (%)
	No alarm	RST	RESET hole
	Manufacturer		Use-by date
	Serial number		USB
	Battery anode		Battery cathode
IP22	International Protection		Fully charged
	Temperature limitation		Humidity limitation
	Atmospheric pressure limitation		This way up
	Fragile, handle with care		Keep away from rain
	Low battery		Close the sound prompt
	Pause the sound prompt		Open the sound prompt
	Menu/ Power button		Close the PR sound
	Recyclable		Open the PR sound
R●	Recording	Sensor Fault	Probe failure
Sensor Off	The probe is disconnected.	Finger Out	The finger is not inserted.
	This item is compliant with Medical Device Directive 93/42/EEC of June 14, 1993, a directive of the European Economic Community.		
	Disposal according to EC Directive 2002/96/EC - WEEE (Waste Electrical and Electronic Equipment)		
	UKCA-Marking		Date of manufacture
	Medical device		

10. Function Specification

Information	Display Mode
The Pulse Oxygen Saturation (SpO ₂)	2-digit digital display
Pulse Rate (PR)	3-digit digital display
Pulse Intensity (bar-graph)	bar-graph display

SpO₂ Parameter Specification	
Measuring range	0%~100%, (the resolution is 1%).
Accuracy	70%~100%: ±2%, Below 70% unspecified.
Average value	Calculate the average value from all 4 measured values. The deviation between average value and true value does not exceed 1%.
Pulse Parameter Specification	
Measuring range	30bpm~250bpm, (the resolution is 1bpm)
Accuracy	±2bpm or ±2% (select larger)
Average pulse rate	Moving calculate the Average pulse rate every 4 cardio-beat's cycle. The deviation between average value and true value does not exceed 1%
Safety Type	Interior Battery, BF Type
Pulse Intensity	
Range	Continuous bar-graph display, the higher display indicates the stronger pulse.
Battery Requirement	
Voltage 3.7 rechargeable lithium battery × 1	
Battery working life	
Charge and discharge no less than 500 times.	
Power Adapter	
Input Voltage	100 to 240 VAC, 50/60 Hz
Output voltage	DC 5 V
Output current	1000mA
Oximeter Probe	
Wavelength:660nm~905nm	
Dimensions and Weight	
Dimensions	61(L) × 56(W) × 24 (H) mm
Weight	About 60g (with the lithium battery*1)

11. Warranty and service

We provide 1 year warranty for material and manufacturing defects of the product.

The warranty does not apply:

- in case of damage caused by improper operation
- for wearing parts
- for defects that were already known to the customer at the time of purchase
- in case of the customer's own fault

The statutory warranties of the customer remain unaffected by the guarantee.

For the assertion of a warranty claim within the warranty period, the customer must provide proof of purchase. The warranty must be claimed within a period of 1 year from the date of purchase to: Novidion GmbH, Fuggerstr. 30, 51149 Cologne, Germany.

The customer has the right to have the goods repaired by us in case of a warranty claim or at workshops authorised by us. The customer is not granted any further rights based on the warranty.

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Appendix 2

Guidance and manufacture's declaration
Guidance and manufacture's declaration – electromagnetic emissions-
for all EQUIPMENT and SYSTEMS

Guidance and manufacture's declaration – electromagnetic emission		
The Pulse Oximeter is intended for use in the electromagnetic environment specified below. The customer of the user of the Pulse Oximeter should assure that it is used in such and environment.		
Emission test	Compliance	Electromagnetic environment – guidance
RF emissions CISPR 11	Group 1	The Pulse Oximeter is suitable for use in all establishments, including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
RF emission CISPR 11	Class B	
Harmonic emissions IEC 61000-3-2	Not applicable	
Voltage fluctuations/flicker emissions IEC 61000-3-3	Not applicable	

Guidance and manufacture's declaration – electromagnetic immunity –
for all EQUIPMENT and SYSTEMS

Guidance and manufacture's declaration – electromagnetic immunity			
The Pulse Oximeter is intended for use in the electromagnetic environment specified below. The customer or the user of Pulse Oximeter should assure that it is used in such an environment.			
Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment - guidance
Electrostatic discharge (ESD) IEC 61000-4-2	±6 kV contact ±8 kV air	±6 kV contact ±6 kV air	Floors should be wood, concrete or ceramic tile. If floor are covered with synthetic material, the relative humidity should be at least 30%. The manufacturer may recommend the ESD precautionary procedures to user.
Power frequency (50/60Hz) magnetic field IEC 61000-4-8	3A/m	3A/m	Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment.

Guidance and manufacture's declaration – electromagnetic immunity –
for EQUIPMENT and SYSTEMS that are not LIFE-SUPPORTING

Guidance and manufacture's declaration – electromagnetic immunity

The Pulse Oximeter is intended for use in the electromagnetic environment specified below. The customer or the user of Pulse Oximeter should assure that it is used in such an environment.

Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment - guidance
Conducted RF IEC 61000-4-6	3 Vrms 150 kHz to 80 MHz	3 Vrms	Portable and mobile RF communications equipment should be used no closer to any part of the Pulse Oximeter, including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter. Recommended separation distance $d = \left[\frac{3.5}{V_1} \right] \sqrt{P}$ $d = \left[\frac{3.5}{E_1} \right] \sqrt{P} \quad 80 \text{ MHz to } 800 \text{ MHz}$ $d = \left[\frac{7}{E_1} \right] \sqrt{P} \quad 800 \text{ MHz to } 2.5 \text{ GHz}$ <p>Where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation distance in meters (m). Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey, should be less than the compliance level in each frequency range. Interference may occur in the vicinity of equipment marked with the following symbol:</p> 
Radiated RF IEC 61000-4-3	3 V/m 80 MHz to 2.5 GHz	3 V/m	

NOTE 1: At 80 MHz and 800 MHz, the higher frequency range applies.

NOTE 2: These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

A Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the Pulse Oximeter is used exceeds the applicable RF compliance level above, the Pulse Oximeter should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as reorienting or relocating the Pulse Oximeter.

b Over the frequency range 150 kHz to 80 MHz, field strengths should be less than 3 V/m.

Recommended separation distances between portable and mobile RF communications equipment and the EQUIPMENT or SYSTEM –
for EQUIPMENT or SYSTEM that are not LIFE-SUPPORTING

Recommended separation distances between portable and mobile RF communications equipment and the Pulse Oximeter.

The Pulse Oximeter is intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The customer or the user of the Pulse Oximeter can help prevent electromagnetic interference

by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the Pulse Oximeter as recommended below, according to the maximum output power of the communications equipment.

Rated maximum output power of transmitter (W)	Separation distance according to frequency of transmitter (m)		
	150 kHz to 80 MHz	80 MHz to 800 MHz	800 MHz to 2.5 GHz
	$d = \left[\frac{3.5}{V_i} \right] \sqrt{P}$	$d = \left[\frac{3.5}{E_i} \right] \sqrt{P}$	$d = \left[\frac{7}{E_i} \right] \sqrt{P}$
0.01	0.117	0.117	0.233
0.1	0.369	0.369	0.738
1	1.167	1.167	2.333
10	3.689	3.689	7.379
100	11.67	11.67	23.33

For transmitters rated at a maximum output power not listed above, the recommended separation distance d in meters (m) can be estimated using the equation applicable to the frequency of the transmitter, where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.

NOTE 1: At 80 MHz and 800 MHz, the separation distance for the higher frequency range applies.

NOTE 2: These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.