Pulsoximeter PO-250 – Bedienungsanleitung  
Pulse Oximeter PO-250 – Instruction manual  
Pulsioxímetro PO-250 – Manual de instrucciones  
Oxymètre de Pouls PO-250 – Mode d'emploi  
Pulsossimetro PO-250 – Istruzioni per l'uso  
Pulsoxymeter PO-250 – Gebruiksaanwijzing  
Pulsoksymetr PO-250 – Instrukcja obsługi  
Pulsoximeter PO-250 – Bruksanvisning

*[www.pulox.de](http://www.pulox.de)*

Fuggerstr. 30 • 51149 Köln • Deutschland  
Tel.: +49 (0) 2203 / 9885 200  
Fax: +49 (0) 2203 / 9885 206  
www.pulox.de • Mail: info@novidion.de

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## Instructions to User

Dear users, thank you very much for purchasing the Pulse Oximeter.

This Manual is written and compiled in accordance with the council directive MDD93/42/EEC for medical devices and harmonized standards. In case of modifications and software upgrades, the information contained in this document is subject to change without notice.

The Manual describes, in accordance with the Pulse Oximeter's features and requirements, main structure, functions, specifications, correct methods for transportation, installation, usage, operation, repair, maintenance and storage, etc. as well as the safety procedures to protect both the user and equipment. Refer to the respective chapters for details.

Please read the User Manual carefully before using this product. The User Manual which describes the operating procedures should be followed strictly. Failure to follow the User Manual may cause measuring abnormality, equipment damage and human injury. The manufacturer is NOT responsible for the safety, reliability and performance issues and any monitoring abnormality, human injury and equipment damage due to users' negligence of the operation instructions. The manufacturer's warranty service does not cover such faults.

Owing to the forthcoming renovation, the specific products you received may not be totally in accordance with the description of this User Manual. We apologize for this.

This product is a medical device, which can be used repeatedly.

### Warning

- ! Uncomfortable or painful feeling may appear if using the device ceaselessly, especially for the microcirculation barrier patients. It is recommended that the sensor should not be applied to the same finger for over 2 hours.
- ! For special patients, there should be a more prudent inspection in the placing process. The device can not be clipped on edema and tender tissue.
- ! The light (infrared is invisible) emitted from the device is harmful to the eyes, so the user and maintenance personnel should not stare at the light.
- ! Testee can not use enamel or other makeup.
- ! Testee's fingernail can not be too long.
- ! Please refer to the correlative literature about the clinical restrictions and caution.
- ! This device is not intended for diagnosis.

The User Manual is published by our company. All rights reserved.

## 1. Safety

### 1.1 Instructions for Safe Operations

- ◇ Check the main unit and all accessories periodically to make sure that there is no visible damage that may affect patient's safety and monitoring performance about cables and transducers. It is recommended that the device should be inspected once a week at least. In case of obvious damage, stop using the device.
- ◇ Necessary maintenance must be performed by qualified service engineers ONLY. There are no user serviceable parts and users are not permitted to maintain it by themselves.
- ◇ The oximeter cannot be used together with devices not specified in the User Manual. Only accessory appointed or recommended by manufacture can be used with this device.
- ◇ This product is calibrated before leaving the factory.

### 1.2 Warning

- ! Explosive hazard—DO NOT use the oximeter in environment with inflammable gas such as some ignitable anesthetic agents.
- ! Ensure that the environment in which the device is operated is not subject to any sources of strong electromagnetic interference, such as radio transmitters, mobile telephones, etc. Keep them far away High level electromagnetic radiation emitted from such devices may greatly affect the monitor performance.
- ! DO NOT use the oximeter while the testee is being measured by MRI and CT.
- ! Be careful with the use of the lanyard cord. Improper use of the lanyard cord will cause device damage not covered under the manufacturer's warranty. Swinging the device by the lanyard cord will void the warranty. Please do not use lanyard cord if allergic to lanyard cord.
- ! The person who is allergic to rubber can not use this device.
- ! The disposal of scrap instrument and its accessories and packings (including battery, plastic bags, foams and paper boxes) should follow the local laws and regulations.
- ! Please check the packing before use to make sure the device and accessories are totally in accordance with the packing list, or else the device may have the possibility of working abnormally.
- ! Please choose the accessories and probe which are approved or manufactured by the manufacturer, or else it may damage the device.
- ! Please don't measure this device with functional tester for the device's related information.
- ! When uploading, do keep the patient from touching the USB port. The computer used when uploading must be in accordance with EN60950. In addition, when the data line connected to a computer, the medical electrical systems should be in accordance with IEC60601-1-1.

### 1.3 Attention

- Keep the oximeter away from dust, vibration, corrosive substances, explosive materials, high temperature and moisture.
- If the oximeter gets wet, please stop operating it.
- When it is carried from cold environment to warm or humid environment, please do not use it immediately.
- DO NOT operate the keys on the front panel with sharp materials.
- High temperature or high pressure steam disinfection of the oximeter is not permitted. Refer to User Manual in the relative chapter for instructions of cleaning and disinfection.
- Do not have the oximeter immersed in liquid. When it needs cleaning, please wipe its surface with medical alcohol on soft tissue. Do not spray any liquid on the device directly.
- When cleaning the device with water, the temperature should be lower than 60°C.
- As to the fingers which are too thin or too cold, it would probably affect the normal measurement of the patients' SpO<sub>2</sub> and pulse rate, please clip a thicker finger such as thumb and middle finger deeply enough into the probe.
- The update period of data is less than 5 seconds, which is changeable according to different individual pulse rate.
- The waveform is normalized. Please read the measured value when the waveform on screen is equably and steady-going. Here this measured value is optimal value, and the waveform at the moment is the standard one.
- If some abnormal conditions appear on the screen during the test process, pull out the finger and reinsert to restore normal use.
- The device has an estimated lifespan of three years from the first electrified use.
- The hanging rope attached to the device is made from Non- allergy material, if particular group are sensitive to the hanging rope, stop using it. In addition, pay attention to the use of the hanging rope, do not wear it around the neck on the purpose of avoiding harm to the patient
- This device has the function of alarming, users can check on this function according to chapter 6.2 as a reference.
- The device has the function of limits alarming, when the measured data is beyond the highest or lowest limit, the device would start alarming automatically on the premise of the alarming function is on.
- The device has the function of alarming, this function can either be paused, or closed (default setting) for good. This function could be turned on through menu operation if you need. Please check the chapter 6.2 as a reference.
- The device may not work for all patients. If you are unable to achieve stable readings, discontinue use.
- A flexible circuit connects the two parts of the device. Do not twist or pull on the connection.

## 2. Overview

The pulse oxygen saturation is the percentage of HbO<sub>2</sub> in the total Hb in the blood, in other words the O<sub>2</sub> concentration in blood. It is an important bio-parameter for respiration. A number of diseases relating to the respiratory system may cause the decrease of SpO<sub>2</sub> in the blood. Serious symptoms might bring danger to person's life. Therefore, prompt information of patients' SpO<sub>2</sub> is of great help for the doctor to discover the potential danger, and is of great importance in the clinical medical field.

The Pulse Oximeter features in small volume, low power consumption, convenient operation and being portable. It is only necessary for patients to put one finger into the probe for diagnosis, and a display screen will directly show the measured value of pulse oxygen saturation.

### 2.1 Classification

Class IIb, (MDD93/42/EEC IX Rule 10)

### 2.2 Features

- A. Operation of the product is simple and convenient.
- B. The product is small in volume, light in weight and convenient for carrying.
- C. Low power consumption

### 2.3 Major Applications and Scope of Application

The Pulse Oximeter can be used in measuring the pulse oxygen saturation and pulse rate through finger. The product is suitable for being used in family, hospital, oxygen bar, community healthcare, physical care in sports (it can be used before or after doing sports and it is not recommended to use the device during the process of exercises) and more.

-  The problem of overrating would emerge when the patient is suffering from toxicosis caused by carbon monoxide, the device is not recommended to be used under this circumstance.

### 2.4 Environment Requirements

Storage Environment

- a. Temperature : -40°C~+60°C
- b. Relative humidity : ≤95%
- c. Atmospheric pressure : 500hPa~1060hPa

Operating Environment

- a. Temperature: +10°C ~+40°C
- b. Relative Humidity : ≤75%
- c. Atmospheric pressure: 700hPa~1060hPa

## 3. Principle

The principle of the Oximeter is as follows: An experience formula of data process is established taking use of Lambert Beer Law according to Spectrum Absorption Characteristics of Reductive Hemoglobin (Hb) and Oxyhemoglobin (HbO<sub>2</sub>) in glow & near-infrared zones. The operation principle of the device is: Photoelectric Oxyhemoglobin Inspection Technology is adopted in accordance with Capacity Pulse Scanning & Recording Technology, so that two beams of different

wavelengths of lights can be focused onto a human nail tip through perspective clamp finger-type sensor. The measured signal can be obtained by a photosensitive element, information acquired through which will be shown on screen through treatment in electronic circuits and microprocessor.

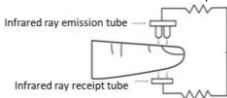
Figure 1 Operating principle

## 4. Technical Specifications

### 4.1 Main Performance

- A. SpO2 value display
- B. Pulse rate value display, bar graph display
- C. Pulse waveform display
- D. Low-voltage indication: low-voltage indicator appears before working abnormally which is due to low-voltage
- E. Display direction can be changed automatically, easy to view.
- F. The product will enter standby mode when no signal is in the product within 5 seconds.
- G. A pulse rate sound indication
- H. With alarm function
- I. With SpO2 value and pulse rate value of storage, the stored data can be uploaded to computers
- J. Data can be transmitted to computers

### 4.2 Main Parameters

#### A. Measurement of SpO2

Measuring range: 0%~100%

Accuracy: 70% ~ 100%,  $\pm 2\%$ ; 0% ~ 69%, unspecified

#### B. Measurement of pulse rate

Measuring range: 30bpm~250bpm

Accuracy:  $\pm 2$  bpm or  $\pm 2\%$  (select larger)

#### C. Resolution

SpO2: 1%, Pulse rate: 1bpm

#### D. Measurement Performance in weak filling condition

SpO2 and pulse rate can be shown correctly when pulse-filling ratio is 0.4%.

SpO2 error is  $\pm 4\%$ , pulse rate error is  $\pm 2$  bpm or  $\pm 2\%$  (select larger).

#### E. Resistance to surrounding light

The deviation between the value measured in the condition of man-made light or indoor natural light and that of darkroom is less than  $\pm 1\%$ .

#### F. Power supply requirement

2.6 V DC ~ 3.6V DC

## G. Optical Sensor

Red light (wavelength is 660nm,6.65mW)

Infrared (wavelength is 905nm, 6.75mW)

## H. Adjustable alarm range

SpO<sub>2</sub>: 0%~100%

Pulse Rate: 0bpm~254bpm

## 5. Installation

### 5.1 View of the front panel

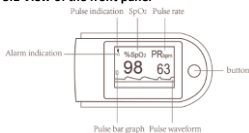
Fig. 2 Front view

Fig. 3 Batteries installation

### 5.2 Battery Installation

Step 1. Refer to Figure 3 and insert the two AAA size batteries properly in the right direction.

Step 2. Replace the cover.

 Please take care when you insert the batteries for the improper insertion may damage the device.

 When you don't use the device for more than 2 hours, please take out the dry batteries.

### 5.3 Accessories

Hanging rope, data line, 2x AAA-batteries, user manual, PC-Software

## 6. Operating Guide

### 6.1 Application Method

#### A. Preparations

- 1) Insert the two batteries properly in the correct direction and then replace the cover.
- 2) Open the clip as shown in Figure 4.

Figure 4 Finger Position

- 3) Put the patient's finger into the rubber cushions of the clip (make sure the finger is in the right position), and then clip the finger.
- 4) Press the switch button once on front panel.

- 5) Do not shake the finger and keep the patient at ease during the process.  
 6) The data can be read directly from the screen on the measuring interface.

 Fingernails and the luminescent tube should be on the same side.

 If the alarm function is on, the device will provide medium-priority alarm signal when finger is out. Intermittent alarm will occur and the user interface presents "FINGER OUT". Medium priority indicating that prompt operator response is required.

#### B. Change display direction:

The display direction of the screen changes automatically according to the handing direction.

#### C. Pause alarm

1. Alarm including the alarm of measure data's going beyond the limits, the alarm of low-voltage, the alarm of finger's out of position.
2. On the measuring interface, if the alarm function is on, during the period of alarming, you can pause it by pressing the button shortly, but the function will be renewed in about 30 seconds.
3. If you want to turn off the alarm permanently, you should enter the menu for operation.

#### D. Data transmission setting

Firstly, please install the affiliated software on the computer, and two icons will appear on the desktop after installation. The icon of "SpO<sub>2</sub> Assistant" is the program for receiving real-time data and stored data which is shown as Figure 5.

- a) Please connect the device to the computer with the affiliated data line, then

double click the "SpO<sub>2</sub> Assistant" icon and click " " and then click "Connect" to start the program.

- b) When you close the "SpO<sub>2</sub> Assistant", there is a dialog box "Save To File" appearing on the desktop, in which you can input some patient's basic information.

Figure 5. SpO<sub>2</sub> program

 When the user chooses to turn on the synchronizing display function on computer, it will probably take several seconds for the data to appear on the computer screen.

#### E. Menu operation

On the measuring interface, the display direction changes according to the handing direction.

Press the button with a prolonged push (1 second) to enter the Settings Menu Interface (see Figure 6).

The user can setup the following parameters in the Settings Menu –Turn on

alarm, turn on pulse sound, alarm high-low limits, data storage (recording). Please note in the Settings Menu:

**CLICK** = short press of button and **PRESS** = prolonged push of button (1sec).

Figure 6 Main Menu Interface

#### a) Alarm setting

On the main menu interface, click the button to select “Alarm”, Press the button (1sec) to enter the alarm setting interface as shown in Figure 7:

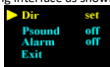
Figure 7

#### a. Adjusting the high and low limits of alarms

On the alarm setting interface, Click the button to select “Dir”, then press the button (1sec) to enter the alarm direction setting interface as shown in Figure 8:

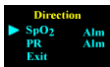
Figure 8

On the alarm direction setting interface, click the button to select “SpO<sub>2</sub> Alm” or “PR Alm”, then Press the button (1 sec) to enter the SpO<sub>2</sub> or PR direction setting interface as shown in Figure 9:

Figure 9.

Click the button to select “Dir”, then Press the button to choose Up or Down (this will be the direction the value of the high-low limits of SpO<sub>2</sub> and pulse rate will be adjusted)

To raise the SpO<sub>2</sub> and pulse rate limit, choose “Dir” as ‘Up’, then Click the button to select high limit (High) or low limit (Low). Press the button and hold to adjust the selected limit to the desired higher value and release the button once the higher limit has been reached.

To lower the SpO<sub>2</sub> and pulse rate limit, choose “Dir” as ‘Down’, then Click the button to select high limit (High) or low limit (Low). Press the button and hold to adjust the selected limit to the desired lower value and release the button once the lower limit has been reached.

If the alarm function is on, the device will provide medium-priority alarm signal when the data of SpO<sub>2</sub> or pulse rate is beyond the limit. Intermittent alarm will occur and the measuring value will be flashing. Medium priority indicating that prompt operator response is required.

**b. Pulse sound indication setting**

In the alarm setting interface as shown in Figure 7, click the button to select "Psound", then press the button to choose to have the Pulse Sound (heart beat) alarm "on" or "off".

**c. Alarm state setting**

In the alarm setting interface as shown in Figure 8, click the button to select "Alarm", then press the button to choose alarm on or off, press "on" to turn on the alarms and "off" to turn off the alarm.

**d. Exit the Alarm settings, the alarm direction setting, the SpO<sub>2</sub> or PR direction setting**

Click the button to select "Exit", then press the button to exit the Menu and return to the previous Menu.

**b) Data storage setting**

This instrument has the ability to store 24 hours worth of data. It can store the measured pulse rate and SpO<sub>2</sub> value accurately, transfer the data to the computer, display the data and print reports.

- a. In the main menu interface as shown in Figure 7, click the button to select "Record", then press the button to choose whether store the data or not, choose "on" to permit storing, choose "off" to forbid storing.
- b. If the data storage function is being turned on, when return to the measuring interface, a flashing yellow dot would appear on screen, which means the device is in a state of storing.
- c. In the state of storing, the device is in measuring interface, the sign "Recording" would appear on the screen in 30 seconds, and then the screen will be automatically shut down, only a flashing yellow dot appears on screen. If short press the button at this moment, the sign "Recording" would appear on the screen, and then the screen will be automatically shut down again; if long press the button, the device would return to the measuring interface.
- d. If turning on the data storage function, the former data storage will be automatically removed.
- e. When the storage space is full, it displays "Memory is full" on the screen, and then shut down in a few seconds. But it will still display "Memory is full" by the next time you turn on the device on the purpose of warning the user, and a few seconds later enter the measuring interface.

**c) Uploading the data to the PC after recording**

- a. Please connect the device to the computer by the data line which is equipped with the device, then double click "SpO<sub>2</sub> Assistant" icon which is shown as Figure 5 to open "SpO<sub>2</sub> Assistant" program, click the 'View device

stored data only', and then click 'Connect'. The Software will then display "Device Stored Data", click the user and click the 'Receive Data' and click the 'OK'.

If you want to watch the data, you can click "" and then choose the data to open.

- b. On the main menu interface, the users can upload the stored data to computer when the symbol "Record" shows "off".

#### d) Exit the main menu

On the main menu interface, click the button to select "Exit", then press the button to exit the main menu.

#### F. Standby mode

If the measuring finger is out of the device in the measuring interface, it will enter standby mode when there is no signal within 5 seconds.

The device can't enter standby mode when it is in a state of storing.

#### 6.2. Attention for Operation

1. Please check the device before using, and confirm that it can work normally.
2. The finger should be placed properly (see the attached illustration of this manual, Figure 4), or else it may cause inaccurate measurement.
3. The SpO2 probe and photoelectric receiving tube should be arranged in a way with the subject's arteriole in a position there between
4. The SpO2 probe should not be used at a location or limb tied with arterial canal or blood pressure cuff or receiving intravenous injection.
5. Do not fix the SpO2 sensor with adhesive or else it may result in venous pulsation and inaccurate measure of SpO2 and pulse rate.
6. Excessive ambient light may affect the measuring result. It includes fluorescent lamps, dual ruby light, infrared heater, direct sunlight and etc.
7. Strenuous action of the subject or extreme electrosurgical interference may also affect the accuracy.
8. Testee can not use enamel or other makeup.
9. Please clean and disinfect the device after operating according to the User Manual (7.1).

#### 6.3. Clinical restrictions

**A.** As the measurements are taken on the basis of arteriole pulse, substantial pulsating blood flow of subject is required. For a subject with weak pulse due to shock, low ambient/body temperature, major bleeding, or use of vascular contracting drug, the SpO2 waveform (PLETH) will decrease. In this case the measurement will be more sensitive to interference.

**B.** For those with a substantial amount of staining dilution drug (such as methylene blue, indigo green and acid indigo blue), or carbon monoxide hemoglobin (COHb), or methionine (Me+Hb) or thiosalicylic hemoglobin, and some with icterus problem, the SpO2 determination by this monitor may be inaccurate.

**C.** The drugs like dopamine, procaine, prilocaine, lidocaine and butacaine may

also be a major factor blamed for serious error of SpO<sub>2</sub> measure.

**D.** As the SpO<sub>2</sub> value serves as a reference value for judgement of anemic anoxia and toxic anoxia, some patients with serious anemia may also report good SpO<sub>2</sub> measurement.

## 7. Cleaning, Transportation and Storage

### 7.1 Cleaning and Disinfecting

When using alcohol wipes to disinfect the device, please air dry or clean it with clean soft cloth.

### 7.2 Maintain

A. Please clean and disinfect the device before using according to the User Manual (7.1).

B. Please change the battery when the screen shows .

C. Take out the battery if leave the equipment unused for long time.

### 7.3 Transportation and Storage

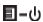
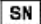

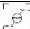
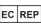
A. The packed device can be transported by ordinary conveyance or according to transport contract. The device can not be transported mixed with toxic, harmful, corrosive material.

B. The packed device should be stored in room with no corrosive gases and good ventilation. Temperature: -40 °C ~ 60 °C; Humidity: ≤ 95%
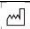
## 8. Troubleshooting

Trouble	Possible Reason	Solution
The SpO <sub>2</sub> and Pulse Rate can not be displayed normally	<ol style="list-style-type: none"> <li>The finger is not properly positioned</li> <li>The patient's SpO<sub>2</sub> is too low to be detected.</li> </ol>	<ol style="list-style-type: none"> <li>Place the finger properly and try again.</li> <li>Try again; Go to a hospital for a diagnosis if you are sure the device works properly.</li> </ol>
The SpO <sub>2</sub> and Pulse Rate are not displayed stably	<ol style="list-style-type: none"> <li>The finger is not placed inside deep enough.</li> <li>The finger is shaking or the patient is moving.</li> </ol>	<ol style="list-style-type: none"> <li>Place the finger properly and try again.</li> <li>Calm thw patient down</li> </ol>
The device can not be turned on	<ol style="list-style-type: none"> <li>The batteries are drained or almost drained.</li> <li>The batteries are not inserted properly.</li> <li>Malfunction of the device.</li> </ol>	<ol style="list-style-type: none"> <li>Change batteries</li> <li>Reinstall batteries.</li> <li>Please contact the local service center.</li> </ol>
The display suddenly turns off	<ol style="list-style-type: none"> <li>The device will power off automatically without signal within 5 seconds.</li> <li>The battery is almost drained.</li> </ol>	<ol style="list-style-type: none"> <li>Turn on the device and insert finger.</li> <li>Change batteries</li> </ol>

## 9. Key of Symbols

Symbol	Description	Symbol	Description
	Type BF applied part		Menu/ Power button
	Refer to instruction manual/booklet		Open the alarm sound indication
%SpO <sub>2</sub>	Pulse oxygen saturation (%)		Open the pulse sound indication
bpm	Pulse rate (bpm)		Serial number
	Low battery		WEEE (2002/96/EC)
	1. No finger inserted 2. Probe error 2. Signal inadequacy indicator		This item is compliant with Medical Device Directive 93/42/EEC of June 14, 1993, a directive of the European Economic Community.
	Battery anode / Battery cathode	IP22	International Protection
	Manufacturer		USB
	This side UP		Date of manufacture
	Fragile, handle with care		Storage and Transport Temperature limitation
	Keep dry		Storage and Transport Humidity limitation
	Recyclable		Storage and Transport Atmospheric pressure limitation
	UKCA marking		European authorised representative
	Swiss authorised representative		UK Authorised Representative

## 10. Function Specification

Information	Display Mode
Pulse Oxygen Saturation (SpO <sub>2</sub> )	2-digit digital LCD display
Pulse Rate (PR)	3-digit digital LCD display
Pulse Intensity (bar-graph)	Bar-graph LCD display
<b>SpO<sub>2</sub> Parameter Specification</b>	
Measuring range	0%~100%, (the resolution is 1%).
Accuracy	70%~100%: ±2%, Below 70% unspecified.
Average value	Calculate the Average value in every 4 measure value. The deviation between average value and true value does not exceed 1%.
<b>Pulse Parameter Specification</b>	

Measuring range	30bpm~250bpm, (the resolution is 1bpm)
Accuracy	±2bpm or±2%, (select larger)
Average pulse rate	The average pulse rate is calculated every 4 pulse cycles. The deviation between the average value and the true value does not exceed 1%.
<b>Safety Type</b>	Interior Battery, BF Type
<b>Pulse Intensity</b>	
Range	Continuous bar-graph display, the higher display indicates the stronger pulse.
<b>Battery Requirement</b>	
1.5V (AAA size) alkaline batteries ×2	
<b>Battery working life</b>	
Two1.5 V (AAA size) 600 mAh alkaline batteries can work continually for 24 hours	
<b>Dimensions and Weight</b>	
Dimensions	58.5(L) × 31(W) × 32 (H) mm
Weight	About 50g (with batteries)

## 11. Warranty and service

We provide 1 year warranty for material and manufacturing defects of the product.

The warranty does not apply:

- in case of damage caused by improper operation
- for wearing parts
- for defects that were already known to the customer at the time of purchase
- in case of the customer's own fault

The statutory warranties of the customer remain unaffected by the guarantee.

For the assertion of a warranty claim within the warranty period, the customer must provide proof of purchase. The warranty must be claimed within a period of 1 year from the date of purchase to Novidion GmbH, Fuggerstr. 30, 51149 Cologne, Germany. In the event of a warranty claim, the customer has the right to have the goods repaired by us or by workshops authorized by us. Further rights are not granted to the customer on the basis of the warranty.

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### **Contec Medical Systems Co., Ltd.**

ADD: No.112 Qinhuang West Street, Economic & Technical Development Zone,  
Qinhuangdao, Hebei Province, PEOPLE'S REPUBLIC OF CHINA

### **Prolinx GmbH**

ADD: Brehmstr. 56, 40239 Duesseldorf, Germany  
Tel: 0049 211 3105 4698  
E-mail: med@eulinx.eu

### **MedPath Limited**

Add: 27 Old Gloucester Street, WC1N 3AX London, United Kingdom  
Tel: +44 776 280 3783, E-mail: UK@medpath.pro  
Web: www.medpath.pro

### **Tinovamed GmbH**

ADD: Zielmattenring 6, 4563 Gerlafingen, Switzerland  
Tel: +41 32 566 71 72  
E-mail: info@tinovamed.ch, www.tinovamed.ch

**Bei Fragen wenden Sie sich bitte an / If you have any questions, please contact /**  
Si tiene alguna pregunta, póngase en contacto con nosotros / Si vous avez une  
question s'il vous plaît contacter / Se si dispone di una domanda si prega di  
contattare / Indien u vragen heeft, kunt u contact opnemen met / W razie  
jakichkolwiek pytań prosimy o kontakt / Om du har några frågor, vänligen  
kontakta:

Novidion GmbH, Fuggerstr. 30, 51149 Köln, Germany  
Tel.: +49 2203 – 9885 200, Fax: +49 2203 – 9885 206  
info@novidion.de, www.pulox.de  
WEEE: DE24355330

**Sollten Sie beim Lesen dieser Anleitung Probleme haben, so können Sie diese  
auch auf unserer Website downloaden / In case you have any problems reading  
this manual, you can download it from our website / En caso de que tenga algún  
problema para leer este manual, puede descargarlo de nuestro sitio web / Si vous  
rencontrez des problèmes pour lire ce manuel, vous pouvez le télécharger depuis  
notre site Web / In caso di problemi nella lettura di questo manuale, è possibile  
scaricarlo dal nostro sito Web / Mocht u problemen hebben met het lezen van**

deze handleiding, dan kunt u die van onze website downloaden / Jeśli masz problemy z odczytaniem tej instrukcji, możesz ją również pobrać z naszej strony internetowej / Om du har problem med att läsa dessa instruktioner kan du också ladda ner dem från vår webbplats:

## <https://www.pulox.de/Anleitungen-Datenblaetter-Retouren>

Version: 1.03 10.11.2023

### Appendix 1

#### Guidance and manufacturer's declaration – electromagnetic emissions- for all EQUIPMENT and SYSTEMS

Guidance and manufacturer's declaration – electromagnetic emission		
The PO-250 is intended for use in the electromagnetic environment specified below. The customer of the user of the PO-250 should assure that it is used in such an environment.		
Emission test	Compliance	Electromagnetic environment – guidance
RF emissions CISPR 11	Group 1	The PO-250 uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.
RF emission CISPR 11	Class B	The PO-250 is suitable for use in all establishments other than domestic and those directly connected to a low voltage power supply network which supplies buildings used for domestic purposes.

#### Guidance and manufacturer's declaration – electromagnetic immunity – for all EQUIPMENT and SYSTEMS

Guidance and manufacturer's declaration – electromagnetic immunity			
The PO-250 is intended for use in the electromagnetic environment specified below. The customer or the user of PO-250 should assure that it is used in such an environment.			
Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment - guidance
Electrostatic discharge (ESD) IEC 61000-4-2	±6 kV contact ±8 kV air	±6 kV contact ±8 kV air	Floors should be wood, concrete or ceramic tile. If floor are covered with synthetic material, the relative humidity should be at least 30%.
Power frequency (50/60Hz) magnetic field IEC 61000-4-8	3A/m	3A/m	Mains power quality should be that of a typical commercial or hospital environment.
NOTE UT is the a.c. mains voltage prior to application of the test level.			

#### Guidance and manufacturer's declaration – electromagnetic immunity – for EQUIPMENT and SYSTEMS that are not LIFE-SUPPORTING

Guidance and manufacturer's declaration – electromagnetic immunity			
The PO-250 is intended for use in the electromagnetic environment specified below. The customer or the user of PO-250 should assure that it is used in such an environment.			
Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment - guidance
Radiated RF IEC 61000-4-3	3 V/m 80 MHz to 2.5 GHz	3 V/m	Portable and mobile RF communications equipment should be used no closer to any part of the PO-250, including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter.

Recommended separation distance

$$d = \left[ \frac{3.5}{V_1} \right] \sqrt{P} \quad d = \left[ \frac{3.5}{E_1} \right] \sqrt{P}$$

80 MHz to 800 MHz

$$d = \left[ \frac{7}{E_1} \right] \sqrt{P}$$

800 MHz to 2.5 GHz

Where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation distance in metres (m).

Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey, a should be less than the compliance level in each frequency range.b

Interference may occur in the vicinity of equipment marked with the following symbol:

NOTE 1: At 80 MHz and 800 MHz, the higher frequency range applies.

NOTE 2: These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

- a. Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the PO-250 is used exceeds the applicable RF compliance level above, the PO-250 should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as reorienting or relocating the PO-250.
- b. Over the frequency range 150 kHz to 80 MHz, field strengths should be less than 3 V/m.

**Recommended separation distances between portable and mobile  
RF communications equipment and the EQUIPMENT or SYSTEM –  
for EQUIPMENT or SYSTEM that are not LIFE-SUPPORTING**

Recommended separation distances between

portable and mobile RF communications equipment and the PO-250

The PO-250 is intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The customer or the user of the PO-250 can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the PO-250 as recommended below, according to the maximum output power of the communications equipment.

Rated maximum output power of transmitter (W)

Separation distance according to frequency of transmitter (m)

	80 MHz to 800 MHz	800 MHz to 2.5 GHz
	$d = \left[ \frac{3.5}{E_1} \right] \sqrt{P}$	$d = \left[ \frac{7}{E_1} \right] \sqrt{P}$
0.01	0.12	0.23
0.1	0.37	0.74
1	1.17	2.33
10	3.69	7.38

100	11.67	23.33
-----	-------	-------

For transmitters rated at a maximum output power not listed above, the recommended separation distance  $d$  in metres (m) can be estimated using the equation applicable to the frequency of the transmitter, where  $P$  is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.

NOTE 1: At 80 MHz and 800 MHz, the separation distance for the higher frequency range applies.

NOTE 2: These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

## Appendix 2

State	Alarm condition delay	Alarm signal generation delay
Low voltage alarm	1s	20ms
SpO2 alarm	330ms	20ms
Pulse rate alarm	330ms	20ms
Probe error alarm	16ms	20ms