


# Intrafix® Primeline Intrafix® SafeSet

# Intrafix® Primeline Flush Intrafix® SafeSet Flush



en	Instructions for use	7
de	Gebrauchsanweisung	8
bg	Инструкция за употреба	9
cs	Návod k použití	10
da	Brugsanvisning	11
el	Οδηγίες χρήσης	12
es	Instrucciones de uso	14
et	Kasutusjuhised	15
fi	Käyttöohje	16
fr	Mode d'emploi	17
hr	Upute za upotrebu	18
hu	Használati utasítás	19
id	Petunjuk penggunaan	21
it	Istruzioni per l'uso	22
ko	사용 지침	23
lt	Naudojimo instrukcija	24

lv	Lietošanas instrukcija	25
nl	Gebruikersinformatie	26
no	Bruksanvisning	27
pl	Instrukcja użytkowania	28
pt	Instruções de utilização	30
ro	Instrucțiuni de utilizare	31
ru	Инструкция по применению	32
sk	Návod na použitie	34
sl	Navodila za uporabo	35
sr	Uputstvo za upotrebu	36
sv	Bruksanvisning	37
th	คำแนะนำในการใช้งาน	38
tr	Kullanım Kilavuzu	39
uk	Інструкції для застосування	40
vi	Hướng dẫn sử dụng	41
zh	使用说明	42



**B | BRAUN**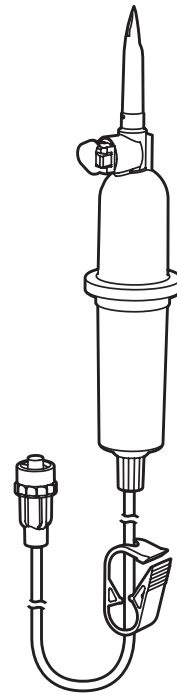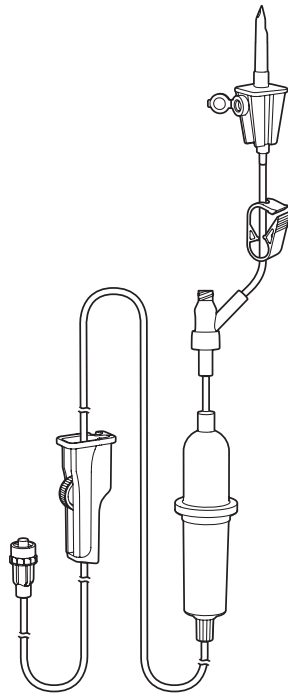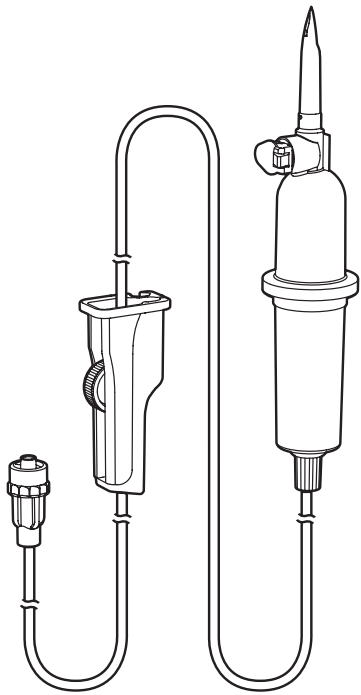


# I. Intrafix® Primeline

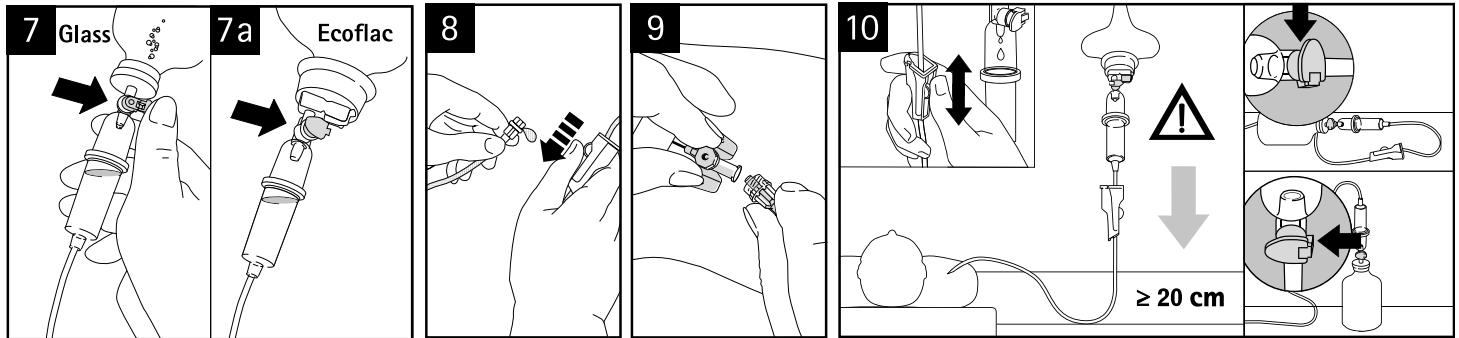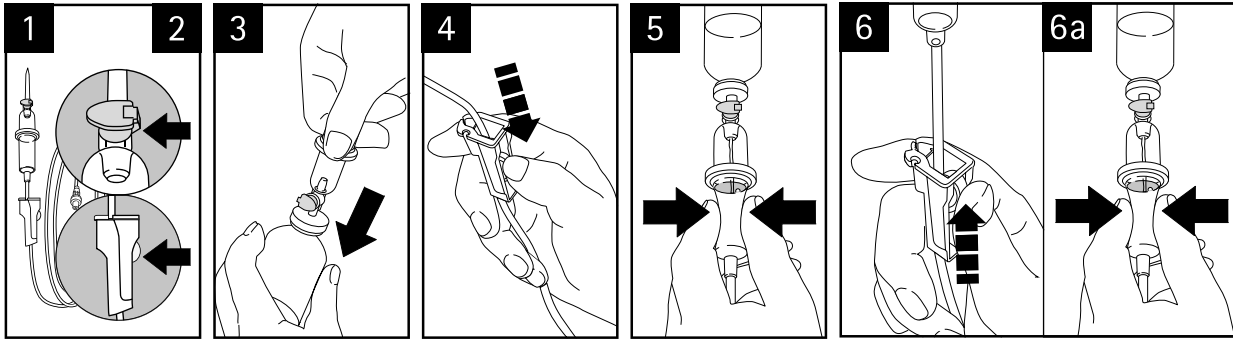
# II. Intrafix® SafeSet
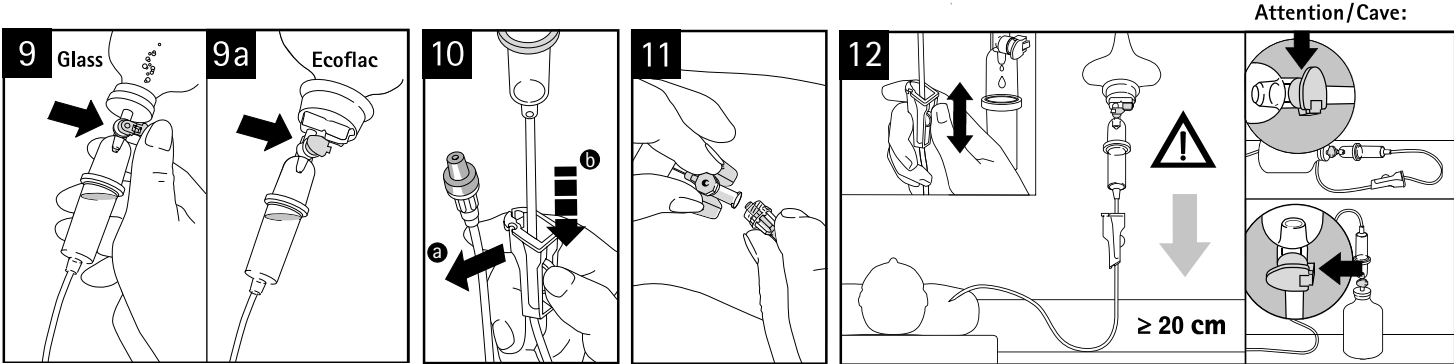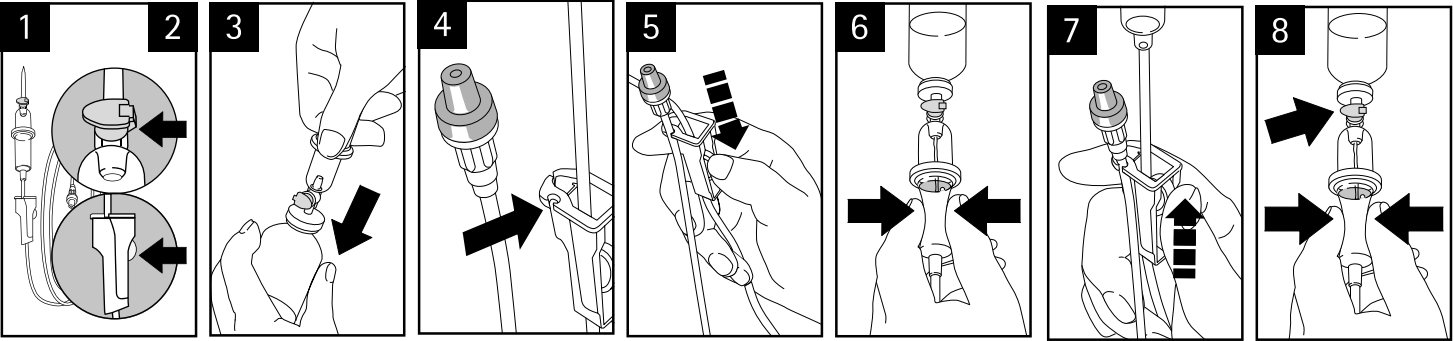
### III. Intrafix® SafeSet Type: Flush with needle free valve
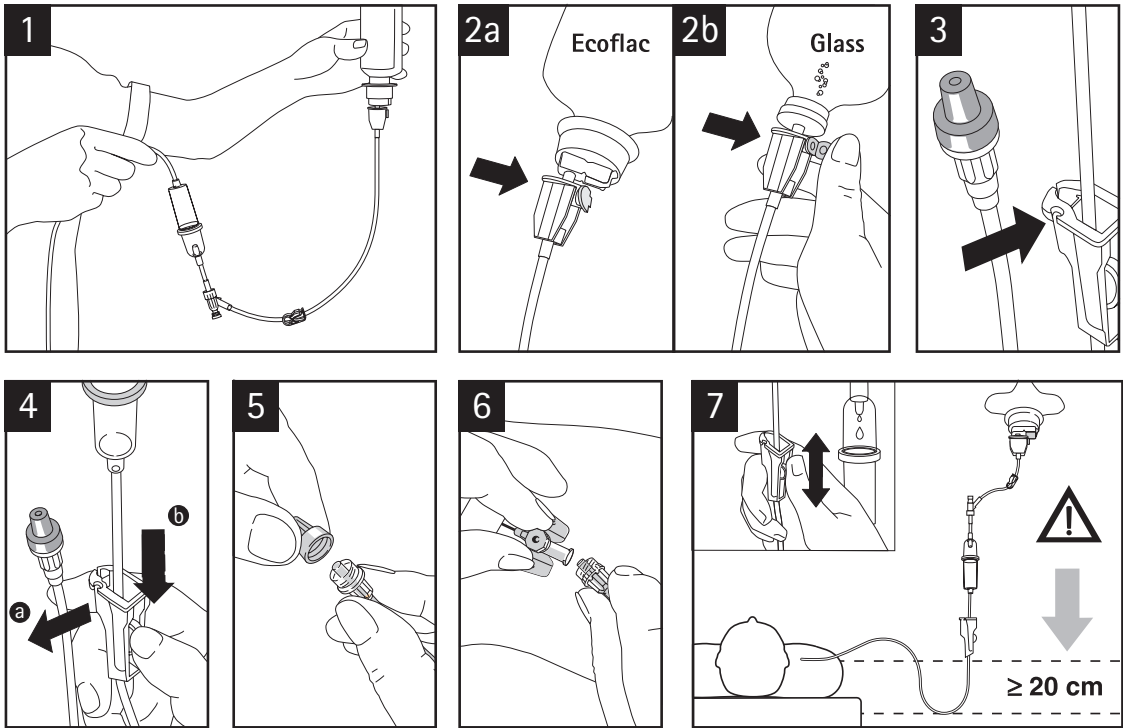
## IV. Intrafix® Primeline: Type Flush secondary line
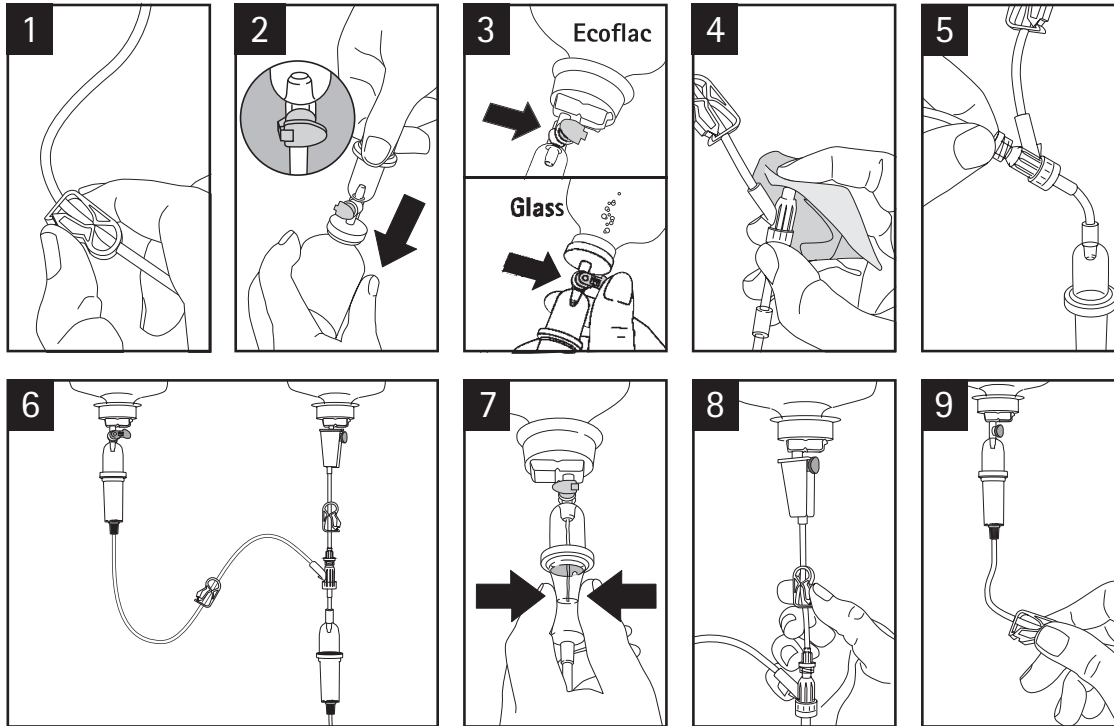
## en Instructions for use

### Device description

Articles equipped with PVC tube:

ID: 3 mm  
OD: 4.1 mm  
Shore A: 74

Articles equipped with PUR-free tube (incl. UV-protected tubes):

ID: 3 mm  
OD: 4.1 mm  
Shore A: 87

IV administration sets can be used with devices which in compliance with EN ISO 80369-7 / ISO 80369-7; EN ISO 80369-20 / ISO 80369-20.

IV administration sets are in accordance with EN ISO 8536-4 / ISO 8536-4 ("G" & "P" devices); EN ISO 8536-8 / ISO 8536-8 ("P" devices)

Always refer to the manufacturer's instruction for use of infusion pumps. (P-Set)

### Sterility

Sterilized using ethylene oxide or irradiation. Please refer to the label of primary packaging

Do not use if package is damaged.

Visually inspect the sterile barrier system for breaches and integrity prior to use.

### Intended purpose

Intrafix® Gravity Sets (G): I.V. administration sets for gravity infusion.

Intrafix® Pressure Sets (P): I.V. administration sets for infusion by gravity and compatible pumps.

Do not re-use.

### Indication

IV administration sets are used for infusion therapy e.g.: Fluid volume maintenance, fluid volume replacement, infusion of total parenteral nutrition, infusion of medications (e.g. antibiotics, chemotherapeutic agents) with bottles and bags according to the SPC (Summary of product characteristics) of the drugs/solutions.

### Patient population

The device can be used for all patients for which infusion therapy is prescribed. No gender or age-related limitations. Intrafix® can be used for adults, pediatric and neonates.

### Intended User

Intrafix® should only be used by healthcare professionals (professional nurses, licensed and/or certified physicians, healthcare workers, midwives, paramedics, pharmacists) who have been adequately trained within the educational background in this technique.

Upon medical assessment and adequate instruction, patients and/or caregivers may also be allowed to take over defined handling steps according to national guidelines. Healthcare workers are responsible for ensuring that patients and/or caregivers are instructed in the correct use of Intrafix® and that the instruction of caregiver and/or patient is documented by the healthcare worker.

### Contraindications

- IV administration set shall not be used in case of proven incompatibility of the drug/IV solution to one of the used materials. SPCs of the drugs/solutions for contraindications should be considered.
- No administration of blood products or blood components.

### Residual Risks/Side effects

General risk occurring during infusion therapy including air embolism, local to systemic infection, particulate contamination. Leakage of drugs/fluids can be an additional hazard to patients, visitors or staff.

### Precautions

Visual check of the IV administration set for damages before use.

Do not use if protective caps are missing or loose.

If pressure infusion (e.g. with a pneumatic cuff) is used, remove air from container beforehand. Observe general safety precautions.

Mind the general guidelines for parallel infusions

Please also consider the corresponding SPCs of the drugs/ solutions regarding possible incompatibilities of drugs/ solutions.

Monitor all connections for tightness over the whole infusion procedure.

Always refer to the manufacturer's instruction for use of infusion pumps. (P-Set)

Due to the structure of the fluid filter of Intrafix® SafeSet which achieves a particle retention with a filter efficiency > 80% for a particle size of 3µm, the infusion of highly viscous solutions, lipid emulsions >10% and certain suspensions can affect the flow rate. With regard to this, please also follow the information in the relevant summary of product characteristics.

Intrafix® Primeline Type: Flush Secondary Line and Intrafix® SafeSet Type: Flush w. needle free valve:

Special IV line for parallel infusion with compatible main line, e.g. Intrafix® SafeSet Flush w. Needle-free valve. (Piggyback). Do not connect an Intrafix® Primeline Flush Secondary line directly to the patient without the main line.

Use aseptic techniques in accordance to national guidelines and/or hospital protocols for variants with Safeflow/ Caresite®.

After medication administration, flush per institutional protocol, attaching completely between flushes, for variants with Caresite®.

### Warning

Do not re-use. Re-use of single-use devices creates a potential risk of patient or user. It may lead to contamination and/or impairment of functional capability. Contamination and/or limited functionality of the device may lead to injury, illness or death of the patient. Administration of light sensitive drugs requires UV-protected IV administration sets. Always check, if incompatibilities with other drugs or with materials is stated in the SPC (Summary of product characteristics) of the drug.

Do not re-attach the protective cap of the patient connector. Do not re-sterilize. Do not use after expiration date.

Variants with Safeflow/ Caresite®:

Only use connection devices in compliance with ISO 80369-7. Syringes or connectors which are not in compliance with ISO 80369-7/ EN ISO 80369-7 can damage the swabable valve. Syringes and male luer connectors have

a large variety of configurations and can vary significantly in design and dimensions.

Do not use, in any situation, needles in the needle-free injection port.

Variants with Discifix®:

Stopcocks may tend to fissure formation by getting in contact with infusion solutions and drugs, alcoholic disinfectants, sudden changes in temperature necessitated by therapy, residues of blood or thrombocytes, precipitates of substances physico-chemically incompatible at the junction points. This may lead to fluid re-flux or air entry. We recommend examining the entire infusion line regarding these hazards. In case of doubt we recommend, especially as to the following drugs, to administer them via a separate (Discifix®), which is resistant against drugs: Ciclosporin, Cyclophosphamid, Etoposid, Lipids, Nimodipin, Phenytoin-sodium, Propofol.

### Operating Instructions

For Intrafix® (Pressure "P") and Intrafix® (Gravity "G") re-spiking according to hospital protocols and / or guidelines is allowed during a single procedure.

### Duration of use

Duration of use is dependent on the intended therapy according to the SPC of the drug or solution. In general, change of IV administration sets should occur according national guidelines (e.g. CDC guideline) and/or hospital protocols. IV administration pressure set (P) should be changed after 24h at the latest in case it is used with an infusion pump.

### Disposal

Disposal acc. to local guidelines and/or clinical protocols.

Use integrated spike protection at flow regulator housing for disposal.

### Storage and handling conditions

Storage conditions:

Keep away from sunlight.

Keep dry.

### Notice to the user

All serious incidents related to this product must be reported to B. Braun and the competent authority in the country where the product is being used.

If further Instructions for Use are required, they can be requested from the manufacturer or can be obtained from the B. Braun homepage: <https://eifu.bbraun.com/>.

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