# **mamivac®**

## ***PUMPSET SUCTION KIT***

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## SUCTION KIT

### BPA FREE

All materials are free of  
bisphenol A (BPA).

As of 2019-01  
GB1010020EN

CE 0124

### Proper use:

Suction kit for the following electric mamivac® breast pumps:

► SENSITIVE-C ► SENSITIVE-CFH ► LACTIVE ► Other piston pumps

For expressing breast milk in case of breastfeeding problems, illness or  
separation of mother and infant.

Please observe safety instructions

Store in dry place

Observe operating instructions

Protect from sunlight

Conformity mark according to  
EU Council Directive 93/42/EEC

Manufacturer

KaWeCo GmbH

Gerlinger Str. 36-38  
71254 Ditzingen  
Germany

Tel.: +49 (0) 71 56 / 1 76 02 - 100  
Fax: +49 (0) 71 56 / 1 76 02 - 500  
info@kaweco.de

mamivac®.com

Made in Germany

Before using the **mamivac®** suction kit, read the operating instructions carefully and keep them for future reference.

## Contents


|     |                                     |    |
|-----|-------------------------------------|----|
| 1.  | Notes on the operating instructions | 12 |
| 2.  | Layout and components               | 13 |
| 3.  | Purpose                             | 14 |
| 3.1 | Indication                          | 14 |
| 3.2 | Contraindication                    | 14 |
| 4.  | Hygiene and cleaning                | 15 |
| 5.  | Practical tips                      | 16 |
| 5.1 | General notes                       | 16 |
| 5.2 | Storing breast milk                 | 16 |
| 5.3 | Accessories                         | 16 |
| 6.  | Technical data                      | 17 |
| 7.  | Disposal                            | 17 |
| 8.  | Spare parts                         | 17 |
| 9.  | Warranty                            | 17 |

## 1. Notes on the operating instructions

These operating instructions are valid for all suction kits of the ECONOMY and PREMIUM series as well as all identically constructed models to be used with **mamivac® SENSITIVE**, **mamivac® SENSITIVE-C**, **mamivac® SENSITIVE-CFH** and **mamivac® LACTIVE**.

|                      |            |                      |               |
|----------------------|------------|----------------------|---------------|
| PREMIUM L            | REF 283120 | ECONOMY L            | REF 283122    |
| Double set PREMIUM L | REF 283123 | Double set ECONOMY L | REF 283124    |
| PREMIUM XL           | REF 283133 | ECONOMY XL           | REF 283128    |
| PREMIUM S            | REF 283129 | PREMIUM XL light     | REF 283121K01 |

## 2. Layout and components

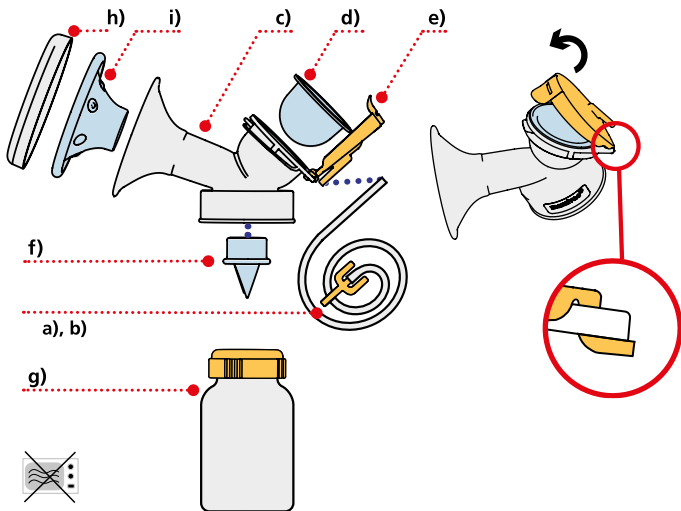- a) Air tube 1 m
- b) Tube connection to the breast pump
- c) Breast shield
- d) Separating diaphragm
- e) Clip-Cover lid
- f) Valve
- g) Milk bottle with screw-on lid
- h) Hygienic cover
- i) Soft insert

Please note that the components of the suction kit vary depending on the model.

The patented protective diaphragm offers the best possible protection and safety for mother and baby. This diaphragm hermetically shuts off the breast pump. Breast milk, bacteria or germs cannot contaminate neither the tubes nor the pump.

You can operate the **mamivac®** breast pumps either with one or two suction kits. To save time, both breasts can be expressed at the same time reducing the expressing time by about 50 %. This gives a mother more time for her baby and family.

**Observe the operating instructions of the breast pump.**

### 3. Purpose

Suction kit to be used with a breast pump for breastfeeding mothers to express breast milk.

#### 3.1 Indication

**mamivac®** suction kits shall be used together with **mamivac®** breast pumps in order to gently and softly express and collect breast milk. Breastfeeding problems can be: Mastitis, anatomic variants of the mamilla (which make latching on difficult), aching or sore mamilla, low or excessive lactation, premature delivery, metabolic disorders of the infant, anatomic misformations of the infant in the facial area, temporary illness of the mother (e.g. medication).

#### 3.2 Contraindication

A possible contraindication is being allergic to the silicone of the breast shield which is in direct contact with the skin. In case of a known silicone allergy, do not use the silicone breast shield. Taking medication in general may affect breastfeeding an infant. The patient information leaflets of the pharmaceutical companies contain appropriate information. In individual cases, the skin may be irritated resulting in redness or red spots.


**mamivac® suction kits may not be used:**

- to suction other body fluids, such as blood, secretions or mucus
- to suction outside the medical field, such as corrosive, flammable or explosive liquids.
- on other living things

## 4. Hygiene and cleaning

Before you use the suction kit for the first time, boil components c) to i) in boiling water for at least 15 min. or use a steam disinfection device, like a sterilizer for instance. Please observe the manufacturer's notices.

- Do not use chemical disinfection baths, solvents or abrasives.
- All parts of the suction kit, except for the air tube and the tube connection, can be cleaned with commercial washing-up liquids by hand or in the dish washer.
- Prior to using the kit the next time, clean the parts which came into contact with milk. It is not necessary to clean the air tube and the tube connection. It is sufficient to wipe off these parts with a damp cloth.
- After cleaning, rinse components c) to i) thoroughly with lukewarm water to remove residues of the washing-up liquid, especially on those parts which come into direct contact with breast milk. You might use a brush to clean areas which are difficult to reach.

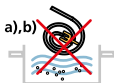**Prior to assembling the pump, make sure that the diaphragm is completely dry and undamaged (no cracks, holes, etc.).**

- Assemble the suction kit according to the drawing in »Layout and components«.
- The suction kit can be used, cleaned and disinfected for at least 50 times.

## 5. Practical tips

### 5.1 General notes

- Prior to expressing, wash your hands thoroughly with soap and wipe the breast with a damp cloth. To avoid dry, soar or cracked nipples, do not use soap or alcohol as detergent.
- Prepare yourself for the expressing with ease. Take a comfortable and relaxed seat. Gently press the breast shield over the nipple for a tight contact. Start to express milk.
- Most pumps have a stimulation mode to support milk production. You can also use your free hand for an additional breast massage.
- Express from both breasts alternately, unless instructed otherwise by the physician. Since expressing times strongly depend on the individual mother, you are asked to contact your midwife or breastfeeding assistant.
- Please check all components for intactness.

### 5.2 Storing breast milk

**Breast milk can be stored as follows:**

- Freshly expressed milk at room temperature (16°-29°C) 3-4 hours
- In the rear of the fridge (below 4 °C) 3 days
- In the freezer (-18 °C) 6 months

Fill the used bottles or containers to two thirds only and write down the fill date. Instead of a bottle, you can also use our specially designed **mamivac®** breast milk freezer bags [REF 282100]. Do not defrost frozen breast milk in the microwave, but at room temperature. Place the bottle or bag in warm water as required. After defrosting, gently shake the container or bag to mix the milk.

### 5.3 Accessories


- **mamivac®** breast milk freezer bags PU = 20 pcs. REF 282100

- **mamivac®** breast shield XL (36 mm)
- **mamivac®** breast shield L (28 mm)
- **mamivac®** breast shield S (20 mm)

REF 283121

REF 283132

REF 283131

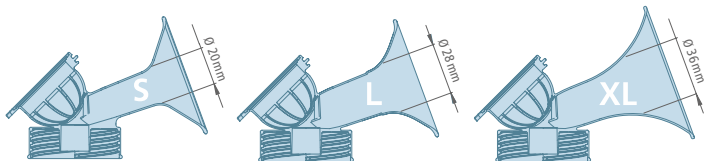## 6. Technical data

| Item                 | REF       | HxWxD (mm)  | Weight | Item                 | REF    | HxWxD (mm) | Weight |
|----------------------|-----------|-------------|--------|----------------------|--------|------------|--------|
| PREMIUM L            | 283120    | 163x118x85  | 220 g  | ECONOMY L            | 283122 | 180x125x90 | 104 g  |
| Double set PREMIUM L | 283123    | 163x170x125 | 438 g  | Double set ECONOMY L | 283124 | 220x160x90 | 217 g  |
| PREMIUM XL           | 283121K01 | 180x125x905 | 120 g  | ECONOMY XL           | 283128 | 180x125x90 | 107 g  |
| PREMIUM S            | 283129    | 163x118x85  | 219 g  | PREMIUM XL           | 283133 | 163x118x85 | 221 g  |

**CE marking in accordance with Council Directive 93/42/EEC: CE 0124**

## 7. Disposal

Please dispose of **mamivac®** suction kits in accordance with your local disposal regulations for plastic products.

## 8. Spare parts

Silicone protective diaphragm

REF 49054124

## 9. Warranty

Ignoring the notices in these operating instructions or an improper use of the device will void any warranty claims. KaWeCo rejects any liability for personal injuries and material damages arising from this.

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