Microdacyn⁶⁰® Wound Care

**Oculus Technologies of México S.A. de C.V.**
Industria Vidriera No. 81,
Industria Zapopan Norte
Zip Code 45130,
Zapopan, Jalisco, México.
Phone: + 52 33 38 33 67 22

**Sonoma Pharmaceuticals Netherlands B.V.**
(Formerly Oculus Innovative Sciences Netherlands B.V.)
Boven de Wolfskuil 3, C30-C32,
6049 LX Herten/Roermond The Netherlands
Phone: +31 (0)475 318 666
Fax: +31 (0)475 318 342
www.sonomapharma.eu

ICDR-003 Rev. 10
June 10th 2021
EOCAC-457 ED-11

Microdacyn⁶⁰® Wound Care

Instructios for use

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Istruzioni d'uso

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Bruksanvisning

Bruksanvisning

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Navodila za uporabo

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Kasutusjuhend

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Instructions for use:

UK

Indications for use: Microdacyn⁶⁰® Wound Care is an electrolysed solution intended for use in the debridement and moistening of surgical wounds (intraoperative and postoperative) and acute and chronic wounds, cuts, abrasions, and burns including those located in a human cavity such as the oral, nasal or ear. Sodium hypochlorite and hypochlorous acid are ancillary substances may have a local antimicrobial effect. Through reducing the microbial load and assisting in creating a moist environment, it enables the body to perform its own healing process. Microdacyn⁶⁰® Wound Care can be broadly applied within a comprehensive wound treatment and is safe on mucous membranes.

Handling: Microdacyn⁶⁰® Wound Care is biocompatible for its indicated use, non-irritating to human skin, eyes or throat, and is not a skin sensitizer. No special handling precautions are required.

Storage: Microdacyn⁶⁰® Wound Care should be stored in its original sealed container at ambient conditions. Store at room temperature and prevent product from freezing. It is non-flammable. After use, the container should be closed again, before storing.

Stability: Stability of the product is assured from date of manufacture until expiration (see product's label for expiration date). Upon opening the stability is assured up to 60 days after opening. Upon opening note the date on the bottle in the section "Date Opened".

Directions for use: Microdacyn⁶⁰® Wound Care is ready for use with no mixing or dilution required. Prior to cleanse the area or use the product, please carried out an appropriate handwashing. Apply Microdacyn⁶⁰® Wound Care directly to the affected area. Use Microdacyn⁶⁰® Wound Care at each wound dressing change. Saturation of the dressing with Microdacyn⁶⁰® Wound Care will assist with dressing removal.

Treatment: Prior to cleanse the area or use the product, please carried out an appropriate handwashing. Soak/saturate dressing with Microdacyn⁶⁰® Wound Care at each dressing change, as and when required. Immersion is allowed for maximum 15 minutes, daily, during initial treatment.

Disposal: The solution is non-toxic and safe to the environment. No special disposal considerations required.

Ingredients: Electrolysed Water, Sodium Chloride, ancillary substances: Sodium Hypochlorite and Hypochlorous Acid.

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