GEBRAUCHSINFORMATION

INSTRUCTIONS FOR USE

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Aktimed TAPE PRO is an elastic self-adhesive tape bandage (kinesiology tape) with a 10% synthetic CBD-solution integrated into the adhesive mass. Contains no THC and therefore does not have an intoxicating effect. The focus is on the physical and mechanical properties of adhesive tape. Effects have not been scientifically proven.

APPLICATION AREAS

- Kinesiologisches Taping

APPLICATION INSTRUCTIONS

- The product can only be used once and needs to be applied by a trained person only.
- Prior to application the skin should be cleaned and fat-free.
- Remove the protection film.
- Apply tape without strain.
- The glue unfolds its adhesive effect through body temperature.
- For this purpose it is advised to smooth out the tape in longitudinal direction from base to end (approx. 60 sec.).
- Apply the tape 30 min prior to sports activity.
- After showering, taking a bath or swimming the tape should be dried with a hair dryer (lowest setting) or by dabbing.
- The tape can occasionally cause itching and skin irritation. In that case please remove the tape.
- The tape can best be removed after taking a shower or a bath.

SAFETY INSTRUCTIONS

- Do not use in case of an allergy against one of the ingredients, or test the tape on a small area of skin first!
- Do not apply on open wounds, injured skin or unhealed scars!
- Do not use in case of acute or chronic skin diseases (e.g. neurodermatitis/eczema, psoriasis, fungal infection)!
- Do not use in case of thrombosis or thrombophlebitis!
- Do not use in case of a peripheral arterial occlusive disease distal to the occlusion!
- Do not use in case of malignant tumors, radiotherapy or chemotherapy!
- If pregnant, please apply the tape only according to your doctor's advice!
- Keep out of reach of children!

DURATION OF USE

The tape lasts up to 6 days on the skin.

TECHNICAL INFO

Size 5 cm x 5 m:

- AT 20201 PZN 16925497 Aktimed TAPE PRO light blue
- AT 20202 PZN 16925711 Aktimed TAPE PRO pink
- AT 20203 PZN 16925734 Aktimed TAPE PRO black
- AT 20204 PZN 16925757 Aktimed TAPE PRO beige
- AT 20205 PZN 16925792 Aktimed TAPE PRO green
- AT 20206 PZN 16938369 Aktimed TAPE PRO dark blue

Supporting material: 95% cotton, 5% spandex
Adhesive composition: synthetic rubber based adhesive mass with 10% synthetic CBD-solution. No THC.

Store in a cool, dry place away from direct sunlight between 5°C and 25°C.

Manufacturer: Aktimed GmbH, Theaterstr. 18,
69117 Heidelberg, Germany
www.aktimed-tape.de

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