INSTRUCTIONS FOR USE

BrightLine Med®

Medical supplies

PRE-FILLED FLUSH SYRINGE

0.9% sodium chloride

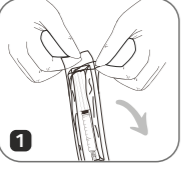
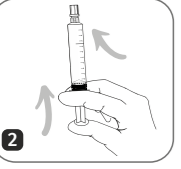
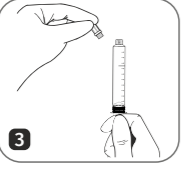
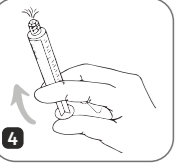
en	PRE-FILLED FLUSH SYRINGE
ar	محقة "إيمريس" المعبئة جاهزة الاستخدام
br	ПРЕДВАРИТЕЛНО НАПЪЛНЕНА СПРИНЦОВКА ЗА ПРОМИВАНЕ
cs	PŘEDPLNĚNÁ PROPLACHOVÁ STŘÍKAČKA
da	FORFYLDT SKYLLESPRØJTE
de	VORGEFÜLLTE SPÜLSPRITZE
el	ΠΡΟΓΕΜΙΣΜΕΝΗ ΣΥΡΙΓΓΙΑ ΕΚΠΛΥΣΗΣ
es	JERINGA DE IRRIGACIÓN PRE-LLENADA
et	EELTÄIDETUD LOPUTUSSÜSTAL
fi	ESITÄYTETTY HUUHTELURUISKU
fr	SERINGUE DE RINÇAGE PRÉ-REMPLE
hr	NAPUNJENA ŠTRCALJKA ZA ISPIRANJE
hu	ELŐRETÖLTÖTT ÖBLÖTŐFESZKENDŐ
it	SIRINGA DI LAVAGGIO PRIEIMPITA
lt	IŠ ANKSTO UŽPILDYTAS PRAPLOVIMO ŠVIKŠTAS
lv	PILNŠĪRICE SKALOŠANAI
mr	SIRINGA TAT-TLAHĪL MIMĪJJA GHAL-LEST
nl	VOORGEVULDE SPOELSPIJT
no	FORHÅNDSFYLT SPYLESPRØYTE
pl	WSTĘPNI NAPĘNIONA STRZYKAWKA DO PRZEPŁUKIWANIA
pt	SERINGA DE IRRIGAÇÃO PRÉ-CHEIA
ro	SERINGĂ DE SPĂLARE PREUMPLUTĂ
sk	PREDPLNĚNÁ PREPLACHOVACIA STRIEKAČKA
sl	NAPOLNJENA BRIZGA ZA IZPIRANJE
sv	FÖRFYLLD SPOLSPRUTA
tr	ÖNCEĐEN DOLDURULMUŞ YIKAMA ŞİRINGASI

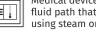
Jiangsu Embrace Science & Technology Development Co., Ltd. 29-2 Yingbin Road, Hua'an Economic and Technological Development Zone, Jiangsu, Zip Code: 223001 www.emstd.cn

Riomavix S.L. Calle de Almansa 55, 1D, Madrid 28039 Spain Tel: +34 658 396 320

Stristes International B.V. Keldemeer 3, 5347 JX Oss The Netherlands

CE 0123

BrightLine Med® Medical supplies	
FOR USE / PROCEDURE	1
	2
	3
	4
	5

		
Consult instructions for use	Caution	Authorized representative in the European Community
		
Date of manufacture	Manufacturer	Use-by-date
		
Keep dry	Keep away from sunlight	Do not re-use
		
Do not use if package is damaged	Lot number	Do not re-sterilize
		
Catalogue number	Medical device contains a sterile fluid path that has been sterilized using steam or dry heat	
CE 0123	CE Mark Conforms to essential requirements of Medical Device Directive 93/42/EEC	

en	PRE-FILLED FLUSH SYRINGE
PRODUCT STRUCTURE:	1. Protective Tip Cap - Luer Lock <p>2. 0.9% sodium chloride</p> 3. Syringe Barrel 4. Piston 5. Plunger Rod

INTENDED USE:
The device is used for closing and flushing the end of catheters at intervals of different dry treatment.

CONTRAINDICATIONS: None

PRECAUTIONS:

- The use of the product must meet the requirements of relevant operational specifications and relevant regulations of medical department, and be restricted to trained doctors or nurses.
- Check for package integrity. It is strictly prohibited to use if product package is damaged, or the syringe breaks, leaks, disintegrates.
- It is strictly prohibited to use if protective tip cap is not installed correctly, loose or falls off.
- Visual examination for clarity of solution, if solution discolouration, turbidity, precipitation or any form of suspended particulate matter material appears - **DO NOT USE.**
- DO NOT RE-STERILIZE**
- Please use the product within the validity period. The product batch number, production date and expiration date are shown on the product packaging label.
- It is strictly prohibited to re-use. Single use only, and discard any unused remaining solution.
- Check the compatibility of 0.9% sodium chloride solution with drugs. Direct contact with drugs incompatible with 0.9% sodium chloride is strictly prohibited.
- It is strictly prohibited to be used for subcutaneous and intramuscular injection and dispensing.
- Only for flushing of intra-vascular catheters. It is not for medication dilutions, nor for dry product reconstitution.
- DO NOT PLACE THE PRODUCT ON A STERILE FIELD.**
- Do not use if you are allergic to the sodium chloride solution.

SHELF LIFE: 3 Years
STERILIZATION DATE: Refer to the package label.
STERILIZATION METHOD: Steam sterilization.
STORAGE: Store at controlled temperature(10-30°C). Do not freeze.

DIRECTIONS FOR USE:
(FOLLOW ASEPTIC TECHNIQUE THROUGHOUT THE PROCEDURE)

- Open syringe package and remove syringe, inspect clarity of solution, check the protective tip cap is in place.
- Push the plunger rod up to release the seal between the piston/plunger and the barrel before removing the protective tip cap.
- Remove protective tip cap by twisting off aseptically.
- Expel air by pushing plunger forward.
- Connect flush syringe to catheter aseptically and push the plunger rod to flush the required volume according to the institution's policy.
- Discard the used syringe and residual part of solution. **DO NOT RE-USE.**

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