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EN Ear Pro " Spray " 20ml / Net 0.68 FL.Oz.
clear liquid

Package Insert " Information for user
Please read entire package insert before use of Ear Pro Waterproofing Ear Spray. Keep this package insert. You may need to refer to it again. If any of the side effects become serious, or if you notice any side effects not listed in this leaflet, cease use and consult your Doctor. Please contact us on **info@ear pro.co**

In this package insert:
1. What is Ear Pro is and what it is used for?
2. Ingredients
3. Warnings: Before you use Ear Pro
4. Directions: How to use Ear Pro?
5. Possible side effects
6. How to store Ear Pro?

1. WHAT IS Ear Pro AND WHAT IT IS USED FOR?

- Ear Pro helps to repel and **prevent water from getting trapped in the ear canal** during water-based activities.
- Ear Pro is suitable for use **by children and adults of all ages.**
- Trapped water in the ear is a quite common problem, and can cause problems such as: discomfort, pain, inflammation and swimmers-ear.
- Ear Pro **creates a water repellent coating** and leaves the ear canal naturally.
- Ear Pro is widely used **by swimmers, families, divers, surfers and competitive athletes.**
- **Otitis is indicated for the prevention of otitis externa** in infants, children, and adults of all ages.

2. Ingredients:

The ingredients in Ear Pro are: **Solution based on paraffin oil and oregano oil.**

3. ⚠️ WARNINGS: Before you use Ear Pro

- Do not use Ear Pro (Contraindications):**
- if you are **allergic (hypersensitive)** to paraffin oil or oregano oil
- if you have a **perforated ear drum or ear infection (e.g. after ear surgery, ventilation tubes)**
- if there is anything else wrong with your ear - such as **pain, swelling, discharge**
- Avoid spraying in eyes, nose or mouth**
- If Ear Pro gets into the eyes**
- Take care to avoid getting Ear Pro in your eyes.
- If this happens, rinse the eye immediately with water for 10 to 15 minutes, holding the eyelids apart and avoiding getting the rinse liquid in the other eye.
- Tell your doctor or pharmacist if irritation persists.
- Pregnancy and breast-feeding**
- Ask your doctor or pharmacist for advice before using it. There is no experience or research on the use of Ear Pro in pregnant or lactating women. Therefore, the product should not be used in case of doubt.
- Ear Pro is for single person use only, don't share.**
- Do not spray near naked flame.**

4. Directions: How to use Ear Pro?

Use Ear Pro only in the ears. Warm in palm

of hand, shake well. Do not point the nozzle at yourself or anyone else.

- Add 1 spray of Ear Pro to each ear
- Softly rub the tragus (outer skin) to make sure the canal is coated.
- Then wipe away any surplus with a tissue.
- Wash hands after use.
- **Do not "plug" the ears with loose cotton wool.**
- **Do not use Ear Pro** when using ear medication or other products placed in the ear.
- **Re-apply after 2 hours** of water exposure
- Always return the bottle to the carton after usage.

5. POSSIBLE SIDE EFFECTS

No side effects are known if the product is used typically only according to the instructions for use. If swallowed side effects may occur and you should consult a doctor.
Stop using if you produce any side effects not mentioned in this leaflet.

6. How to store Ear Pro?

- Keep out of the reach and sight of children.
- Do not use after the expiry date shown on the bottle and the carton. The expiry date refers to the last day of that month.
- Once opened, do not keep the bottle for longer than 6 months.
- Do not store above 45°C / 113°F.
- Store the bottle upright in the outer carton to avoid spillage.

Report any serious incidents to the manufacturer (**info@earpro.co**) and your local medical device Competent Authority

Questions or comments:
www.earpro.co

MD LiquidPro Ltd
166 Old Bakery Street
Valletta VLT1455 Malta

Rev: 20th of February 2025 V1.2

Ear Pro is a registered trademark of LiquidPro Ltd.

Pat. No. EP3795144